GB
CZ
SK
SE
NO
FI
EE
LV

USER INSTRUCTIONS

WIDEX CHARGE n CLEAN CHARGER

WPC301 model

WIDEX

SOUND LIKE NO OTHER

| | | |
|----|--|-----|
| GB | USER INSTRUCTIONS | 3 |
| CZ | NÁVOD K POUŽITÍ | 29 |
| SK | NÁVOD NA POUŽITIE | 54 |
| SE | BRUKSANVISNING | 80 |
| NO | BRUKSANVISNING | 105 |
| FI | KÄYTTÖOHJE | 130 |
| EE | KASUTUSJUHEND | 156 |
| LV | LIETOTĀJA INSTRUKCIJA | 181 |
| | FCC AND ISED STATEMENTS | 207 |
| | COPYRIGHT WS AUDIOLOGY DENMARK A/S | 210 |

USER INSTRUCTIONS

| | |
|--|-----------|
| IMPORTANT SAFETY INFORMATION | 5 |
| YOUR CHARGER | 8 |
| Welcome..... | 8 |
| The charger at a glance..... | 9 |
| Operating, storage and transport conditions..... | 10 |
| OPERATING THE CHARGER | 11 |
| Getting ready for use..... | 11 |
| Power supply..... | 11 |
| Power on..... | 12 |
| Charging the hearing aids..... | 13 |
| Charging indicators..... | 14 |
| After charging..... | 16 |
| Cleaning and drying..... | 17 |
| Overview of light indicators..... | 18 |
| Charging light indicators..... | 18 |
| Maintenance..... | 20 |
| Cleaning..... | 21 |
| TROUBLESHOOTING | 22 |
| REGULATORY INFORMATION | 25 |
| Intended use..... | 25 |

| | |
|-------------------------------------|-----------|
| EU directives..... | 25 |
| Directive 2014/53/EU..... | 25 |
| Information regarding disposal..... | 26 |
| SYMBOLS..... | 27 |

IMPORTANT SAFETY INFORMATION

Read these pages carefully before you begin using your hearing aids.

WARNING

Risk of electric shock!

Connect the charger only with a USB power supply which complies with EN 62368-1.

WARNING

- Observe the operating conditions.
- Pay special attention to protect your device from extreme heat (e.g. fire, microwave oven, induction oven or other high induction fields) and sunlight.
- Charge your hearing instruments only with the approved charger. Ask your hearing care professional for support.
- Do not disassemble the device.
- Do not use the device in explosive atmospheres.
- Do not use devices that are deformed or obviously damaged.
- Do not use the device when its performance after charging reduces considerably. Return the device to your Hearing Care Professional.
- Do not dispose of the device in fire or water.

WARNING

Choking hazard!

Keep the charger and other small parts away from children, and mentally disabled people.

WARNING
Risk of explosion!

Do not use your device in explosive atmospheres (e.g. mining areas).

WARNING
Risk of injury!

Do not use obviously damaged devices and return them to point of sale.

WARNING

Note that any unauthorized changes to the product may cause damage to the product or cause injury.

- Use only approved parts and accessories. Ask your hearing care professional for support.
- Do not modify this product without authorization from the manufacturer.
- If this equipment is modified, appropriate inspection and testing must be conducted to ensure safe use of the product.

WARNING
Risk of eye injury!

In the unlikely event that the ultraviolet (UV) light stays on when the charger lid is opened, do not look directly at the UV light and return the charger to your point of sale.

WARNING

Charging systems may interfere with measuring devices and electronic equipment. The charger complies with applicable standards. Therefore use of the charger in hospitals should be permissible. Nevertheless the charger may interfere with highly sensitive devices. In those cases appropriate steps shall be taken (increase distance from sensitive devices or disconnect the charger from power supply).

NOTICE

- Use of incompatible parts poses a risk of malfunction.
- Use standard USB power adapter which supports an output voltage of 5 V DC and an output current of 0.5 to 1 A.
- Use only the USB cable that is provided with the device.
- Do not insert incompatible or foreign objects into the charger port
- Protect your devices from extreme heat. Do not expose them to direct sunlight.
- Protect your devices from high humidity.
- Do not dry your devices in the micro-wave oven.
- Do not drop. Internal components may get damaged.

YOUR CHARGER

Welcome

Thank you for choosing this product. Your new CHARGE n CLEAN charger is intended to charge your ALLURE hearing aids with built-in power cells (lithium-ion rechargeable batteries), and to ensure their maintenance by means of UVC cleaning, and drying of their external parts (tubes, ear pieces and molds).


Your hearing care professional will advise you on the compatible models. Read these instructions carefully before you start using this device.

NOTICE

Your charger may not look exactly like the one illustrated in this booklet. We reserve the right to make any changes we consider necessary.

The charger at a glance

The charger consists of a charging unit for each hearing aid, a maintenance compartment for the receivers and light indicators that show the charging status of the hearing aids and the status of the maintenance cycle.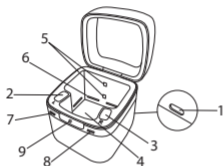

1. USB-C port
2. Left hearing aid charging well
3. Right hearing aid charging well
4. Maintenance compartment for the receivers
5. UVC lights
6. Vent holes
7. Left hearing aid charging light indicators
8. Right hearing aid charging light indicators
9. Maintenance light indicator

Operating, storage and transport conditions

Your charger is designed for indoor use. It is sensitive to extreme conditions such as high temperatures, and it should not be operated, stored or transported in direct sunlight.

Operate the charger within the following conditions:

| | Minimum | Maximum |
|-------------|----------------|----------------|
| Temperature | 10 °C (50 °F) | 35 °C (95 °F) |
| Humidity | 20% rH | 75% rH |

Store the charger within the following conditions:

| | Minimum | Maximum |
|-------------|----------------|----------------|
| Temperature | 10 °C (50 °F) | 40 °C (104 °F) |
| Humidity | 10% rH | 80% rH |

Transport conditions:

| | Minimum | Maximum |
|-------------|----------------|----------------|
| Temperature | -20 °C (-4 °F) | 60 °C (140 °F) |
| Humidity | 5% rH | 90% rH |


You can find technical data sheets and additional information on your charger on www.widex.com.

OPERATING THE CHARGER


Getting ready for use

Power supply

You can connect your charger to different power supplies with 5 V DC, minimum 500 mA output: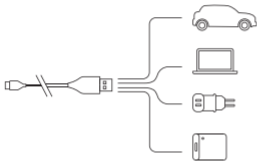

Use only the included USB cable.

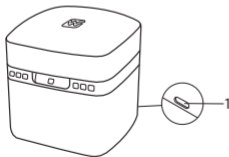1. USB-C port


Connect the cable to the charger:

1. USB-C port

Power on

When the charger is connected to the power supply, the light indicators **will blink green in sequence once** and then switch off: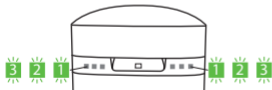

The charger is now in standby mode and ready to charge your hearing aids.

Charging the hearing aids

Gently place your hearing aids in the charging wells and close the cover. The hearing aids switch off automatically.

When connected to power, the charger can charge two hearing aids simultaneously; however, each hearing aid will be charged individually, depending on its own charge level.

This means that, if a fully charged hearing aid is placed in the charger, the charger will not charge.

However, if the other hearing aid is *not* fully charged, the charger will start charging.

Charging your hearing aids fully will take approximately 4 hours. If you charge your hearing aids for half an hour, you will have approximately 4 hours of use.

While charging the hearing aids, the maintenance (cleaning by Ultraviolet-C treatment and drying) is carried out. The cycle takes 90 minutes.

Charging indicators


Once the charger has detected the hearing aids, the charging will begin. One light indicator will **pulsate green** to indicate that the corresponding hearing aid is low on battery and less than **33% charged**:

One light indicator will **glow green** and one will **pulsate green** to indicate that the corresponding hearing aid is more than **33% charged**:

Two light indicators will **glow green** and the third will **pulsate green** to indicate that the corresponding hearing aid is more than **67% charged**: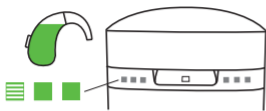

All three light indicators will **glow green** to indicate that the corresponding hearing aid is **100% charged**:

After charging

Avoid removing the hearing aids from the charging wells by pulling the earwire, as this could damage the connection between the earwire and the hearing aid. Instead, hold the hearing aid when removing it from the charger.


When you remove the hearing aid from the charger, the hearing aid switches on after a few seconds. Remember to switch it off if you are not going to use it right away.

Immediately after charging, the hearing aid may temporarily feel warm. If you are not comfortable with this, wait a few minutes for the hearing aid to reach room temperature, before putting the hearing aid on.

Cleaning and drying

NOTICE

Drying and cleaning focus on external parts of hearing aids such as tubes, ear pieces and molds.

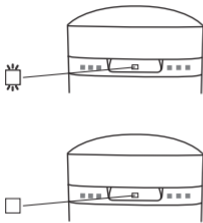Put the hearing aids into the charging slots and close the lid. The light indicator will pulsate **white** to indicate that the maintenance (UVC cleaning* and drying) cycle is in progress:
The light indicator turns **glowing white** to indicate the end of the maintenance cycle:

If the lid is opened during the cleaning or the drying cycle, the maintenance is suspended and the light indicator glows **orange**. The maintenance cycle will restart when the lid is closed.

* Ultraviolet-C light, used for hygienic cleansing. Reduces significantly the risk of infection caused by germs, bacteria and fungi.

Maintenance paused
(**glowing orange**
light)

Overview of light indicators

Charging light indicators

The charge level of both hearing aids is monitored separately, so their status may be different.

Light indicators

Status

All indicators will blink green in a sequence once and then turn off

The charger is connected to power.

One single indicator will pulsate green

The hearing aid is charging.
Low battery, less than 33%

Light indicators

Status

One indicator will glow green and one will pulsate green

The hearing aid is charging.
Battery more than 33%

Two indicators will glow green and the third will pulsate green

The hearing aid is charging.
Battery more than 67%

All indicators will glow green

The hearing aid is fully charged.
Battery 100% charged

Indicators will blink red three times every 2 seconds

A hearing aid error occurred during charging and charging has stopped.
See Troubleshooting

All indicators will blink red in a sequence

A charger error occurred during charging and charging has stopped.
See Troubleshooting

Light indicators


Status

All indicators will blink orange in a sequence

The temperature of the hearing aids has become too high during charging and charging has been paused. See Troubleshooting

Maintenance

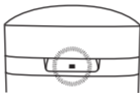Indicator lights

Status

Blinking white

The maintenance cycle is in progress (Ultraviolet-C treatment or drying).

Glowing white

The maintenance cycle is complete.

Glowing orange

The maintenance cycle is paused but the charging cycle continues.

Indicator lights

Status

Blinking red

An error occurred during the maintenance and the cycle has stopped.
See Troubleshooting.

Blinking orange

The maintenance cycle is paused due to over-heating.
See Troubleshooting.

Cleaning

If cleaning is required, disconnect the charger from the power supply and carefully wipe off the charger with a soft, dry cloth.

If you notice any dust in the charging well, gently blow it clean. If this is not enough, gently clean the charging well with a cotton bud.

To remove earwax from the charger, carefully wipe with a soft damp cloth.

After use and before placing in the charger, wipe the charging pins on the hearing aids with a soft, dry tissue to remove grease or moisture.

Risk of injuries caused by electric shock!

If cleaning is required, disconnect the charger from the power supply.

Never use sharp/pointy objects to clean the charger.

TROUBLESHOOTING

These pages contain advice on what to do if your charger does not work as expected. Each hearing aid is monitored separately, so their status may be different

| Light indicators | Possible cause | Solution |
|---|---|--|
| No light indicators | The charger is not connected | Make sure that the USB cable is connected to a power adapter and then plugged into a wall socket |
| | The hearing aid is not placed correctly in the charger | Place the hearing aid correctly in the charger |
| Three or six indicators are blinking orange in a sequence   | The temperature of your surroundings or of the hearing aid has exceeded a critical limit and the charging has been paused | The charging will continue when the temperature of your surroundings or of the hearing aid has dropped. If this continues, find a cooler place to charge your hearing aid |

| Light indicators | Possible cause | Solution |
|---|--|--|
| Three or six indicators are blinking red every 2 seconds  | The charger cannot recognise the hearing aid The hearing aid has a critical error | Remove the incompatible hearing aid(s) Remove the hearing aid, disconnect from the power supply for 10 seconds and then try again. If the charger is still blinking red, contact your hearing care professional |
| All indicators are blinking red in a sequence  | The charger has a critical error, or ambient temperature is below 0 °C | Remove the hearing aid, disconnect from the power supply for 10 seconds and then try again. If the charger is still blinking red, contact your hearing care professional |

| Light indicators | Possible cause | Solution |
|---|--|---|
| Central indicator blinking red  | The maintenance cycle presents a critical error | Remove the hearing aid, disconnect from the power supply for 10 seconds and then try again. If the central indicator is still blinking red, contact your hearing care profes- sional |

REGULATORY INFORMATION

Intended use

This Charger is intended to charge our specific hearing instruments with built-in power cells (lithium-ion rechargeable batteries).

NOTICE

The charging device is intended to be used only indoors and within the defined environmental conditions.

EU directives

Directive 2014/53/EU

Hereby, WSAUD A/S declares that the WPC301 is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU.

A copy of the Declaration of Conformity according to 2014/53/EU can be found at

www.widex.com/doc

Information regarding disposal

Do not dispose of hearing aids, hearing aid accessories and chargers with ordinary household waste.

Hearing aids, hearing aid accessories and chargers should be disposed of at sites intended for electrical waste and electronic equipment, or given to your hearing care professional for safe disposal.

For returning or shipping the devices, please contact your hearing care professional.

Shipment of charger and hearing instruments should follow to local regulations.

SYMBOLS

Symbols commonly used by WSAUD A/S in medical device labelling (labels/IFU/etc.)

| Symbol | Title/Description |
|---|---|
|  | Manufacturer The product is produced by the manufacturer whose name and address are stated next to the symbol. If appropriate, the date of manufacture may also be stated. |
|  | Catalogue number The product's catalogue (item) number. |
|  | Consult instructions for use The user instructions contain important cautionary information (warnings/precautions) and must be read before using the product. |
|  | Warning Text marked with a warning symbol must be read before using the product. |
|  | WEEE mark “Not for general waste”. When a product is to be discarded, it must be sent to a designated collection point for recycling and recovering to prevent the risk of harm to the environment or human health as a result of the presence of hazardous substances. |

| Symbol | Title/Description |
|--|--|
|  | CE mark The product is in conformity with the requirements set out in European CE marking directives. |
|  | RCM mark The product complies with electrical safety, EMC and radio spectrum regulatory requirements for products supplied to the Australian or New Zealand markets. |
|  | FCC logo The product is compliant with relevant USA 47 CFR Rules. FCC – Federal Communications Commission. |
|  | Medical device Indication that the device is a medical device. |

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FCC AND ISED STATEMENTS

Federal Communications Commission Statement

This device complies with part 15/part 18 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.

- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Changes or modifications to the equipment not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

ISED Statement / Déclaration d'ISED

CAN RSS-216/CNR-216

CAN ICES-1/NMB-1

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

- (1) This device may not cause interference.
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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ISED RADIATION EXPOSURE STATEMENT:

This equipment complies with ISED RF radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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COPYRIGHT WS AUDIOLOGY DENMARK A/S

Copyright © WS Audiology Denmark A/S. All rights reserved. Any unauthorized redistribution or use of the software in source or binary form is prohibited and will be prosecuted when discovered.

WSAUD A/S

Nymoellevej 6, DK-3540 Lyngø, Denmark
www.widex.com

Manual no.:
9 514 1290 091 02
Issue:
2025-02