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Gigaset W570 (Widex) – Premium cordless phone with direct transmission to Widex hearing aids

The Gigaset W570 is a premium large button cordless landline phone that transmits phone calls wirelessly and in real-time directly to Widex hearing systems. No supplementary devices are required. You can hear the person you are speaking to directly over both hearing systems and also have best possible speech comprehension when making calls. Simplicity of use and modern design are standout features of the Gigaset W570. It is used like a standard phone and works without any conversions also without hearing systems.

Features

- Makes phone calls stress-free yet with best speech comprehension thanks to the direct and simultaneous transmission of phone conversations into both hearing systems without additional devices.
- Automatic coupling by means of AutoConnect function. The phone automatically establishes a wireless connection to the hearing aids as soon as it is located near hearing systems. The energy-efficient WidexLink wireless technology is deployed. No additional power is drawn from the hearing aid batteries.
- Transmission in real-time. You can hold a conversation immediately after you accept a call.
- Compatible with all Widex hearing systems based upon E-chip platform EVOKE (or newer) and featuring the Widex-Link technology.
- Premium large button cordless phone that can also be used by family members and guests who have normal hearing. This means no additional phone is required in the household.
- Comprehensive functions of the base station, e.g.
 - Memory capacity for up to 150 phone numbers
 - Answer machine with user function
 - Handsfree function with volume control
 - Emergency call function
- Range up to 300 m from the base station (depending on local conditions)

Operation

Coupling the phone and a Widex hearing system

- ▶ Keep the phone handset near your hearing system. The phone is automatically coupled to the hearing system.

Cancelling a coupling

- ▶ Move the phone to outside the range of the hearing systems.

Changing the volume

You can adjust the volume if the sound transmission or WidexLink transmission is too quiet or too loud.

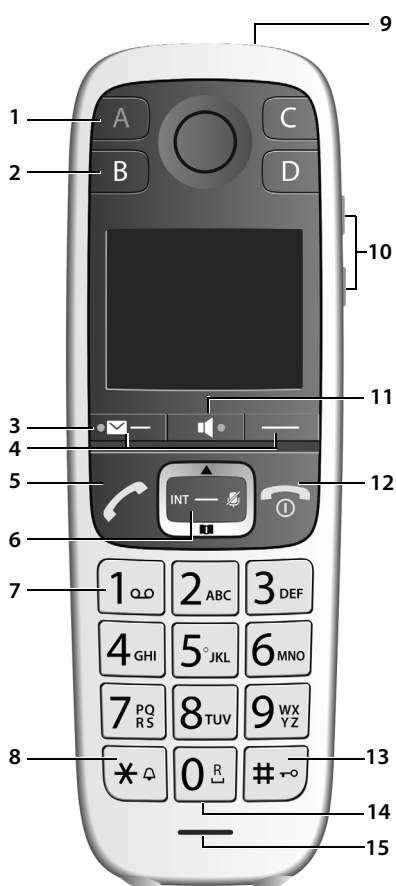
- ▶ Press the volume keys on the right of the handset: at top : louder / at bottom : quieter.

Otherwise the phone works as described in these instructions.

Handset

Overview

Handset

1 **Direct dialling key A** (→ p. 12)

("SOS key", → p. 24)

Illuminated: Emergency function

Flashes: Emergency call started

2 **Direct dialling keys B to D** (→ p. 12)

3 **Message key** (→ p. 21)

Access to the call and message lists;

Flashes: new voice message or new missing call

4 **Display keys** (→ p. 13)

Various functions, depending on the operating situation

5 **Talk key**

Accepting call; selecting displayed number; opening redial list

Flashes: Accepting a call

6 **Control key** (→ p. 11)

Opening a menu; navigating in menus and entry fields; accessing functions (depending on the situation)

7 **Key 1**

Call the network mailbox (→ p. 27)

8 **Star key**

Ringers on/off (press and **hold**) (→ p. 37)

9 **Torch and signal light (LED)** (→ p. 31)

10 **Volume keys**

for ringtone, receiver/headset, speaker

Upper key: Setting "extra-loud" during a call (→ p. 18)

11 **Handsfree key** (→ p. 17)

Switch between earpiece and handsfree mode; accepting call; selecting displayed number; opening redial list

Lights up: handsfree activated

12 **End call key, On/Off key**

End call; cancel function; go back one menu level (press **briefly**); back to idle status (press and **hold**); activate/deactivate handset (press and **hold** in idle status)

13 **Hash key**

Keypad lock on/off (press and **hold**, → p. 11); upper/lower case and digits (→ p. 15)

14 **Recall key**

Consultation call (flash): press and **hold**

15 **Microphone**

Changing the display language: → p. 35

Gigaset W570 base

Registration/paging key:

- Press **briefly**: search for handsets (paging)
→ p. 34
- Press and **hold**: register handsets and DECT devices → p. 33.

Contents

Contents

Overview	2
Handset	2
Gigaset W570 base	3
Illustration in the user guide	5
Safety precautions	6
Getting started	7
Connecting the base	7
Handset	8
Using the phone	11
Getting to know your telephone	11
Making calls	16
Lists	19
Directory	22
Emergency function	24
Using the network mailbox	27
Further functions	28
ECO DECT	28
Using the handset as an alarm clock	29
Baby monitor	30
One touch call	31
Torch/Flashlight	31
Protection against unwanted calls	32
Using multiple handsets	33
Adjusting the telephone	35
Handset settings	35
Base settings	39
Operating the base on the PABX	40
Appendix	41
Customer Service & Assistance	41
Manufacturer's advice	44
Specifications	47
Index	49

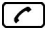
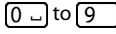
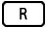
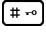
Not all functions described in the user guide are available in all countries or from all network providers.

Illustration in the user guide

	Warnings, which, if not heeded, can result in injury to persons or damage to devices.
	Important information regarding function and appropriate handling or functions that could generate costs.
	Prerequisite for being able to carry out the following action.
	Additional helpful information.

Keys

	Talk key		Handsfree key
	End call key		Number / letter keys
	Control key rim / centre		Message key
	Recall key		Star key
	Hash key		
OK, Back ...		Display keys	

Procedures

Example: Switching Auto answer on/off:

▶ Menu ▶  Settings ▶ OK ▶  Telephony ▶ OK ▶ Auto Answer ▶ OK (= on)

Step	Follow this procedure
▶ Menu	Press the display key Menu when the handset is in idle status. The main menu opens.
▶  ▶ OK	Use the control key  to navigate to the entry Settings . Select OK to confirm. The submenu Settings opens.
▶  Telephony ▶ OK	Select the Telephony entry using the control key  . Select OK to confirm. The submenu Telephony opens.
▶ Auto Answer	The function to switch Auto answer on/off appears as the first menu item.
▶ OK	Select OK to activate or deactivate. Function is activated <input checked="" type="checkbox"/> / deactivated <input type="checkbox"/> .

Safety precautions

Safety precautions

	Read the safety precautions and the user guide before use.
	The device cannot be used in the event of a power failure. It is also not possible to transmit emergency calls . Emergency numbers cannot be dialled if the keypad/display lock is activated!
	Use only rechargeable batteries that correspond to the specification (see list of permitted batteries → www.gigaset.com/service). Never use a conventional (non-rechargeable) battery or other battery types as this could result in significant health risks and personal injury. Rechargeable batteries, which are noticeably damaged, must be replaced.
	The handset must not be operated if the battery cover is open.
	Do not use the devices in environments with a potential explosion hazard (e.g., paint shops).
	The devices are not splashproof. For this reason do not install them in a damp environment such as bathrooms or shower rooms.
	Use only the power adapter indicated on the device. Whilst charging, the power socket must be easily accessible.
	Remove faulty devices from use or have them repaired by our Service team, as these could interfere with other wireless services.
	Do not use the device if the display is cracked or broken. Broken glass or plastic can cause injury to hands and face. Send the device to our Service department to be repaired.
	Do not hold the rear of the handset to your ear when it is ringing or when speaker mode is activated. Otherwise you risk serious and permanent damage to your hearing. The phone may cause interference in analogue hearing aids (humming or whistling) or cause them to overload. If you require assistance, please contact the hearing aid supplier.
	Keep small cells and batteries, which can be swallowed, out of the reach of children. Swallowing a battery can lead to burns, perforation of soft tissue and death. Severe burns can occur within 2 hours of swallowing. In the case of a swallowed cell or battery, seek medical care immediately.
	Using your telephone may affect nearby medical equipment. Be aware of the technical conditions in your particular environment, e.g., doctor's surgery. If you use a medical device (e.g., a pacemaker), please contact the device manufacturer. They will be able to advise you regarding the susceptibility of the device to external sources of high frequency energy (for the Technical data of your Gigaset product see "Technical data").

Getting started

Contents of the package

- One Gigaset W570 base
- One Gigaset handset Gigaset W570H
- Two batteries
- One battery cover
- One power adapter for the base
- An RJ11-RJ11 phone connector cable, such as for connecting to a phone jack on your router
- One user guide

If you have purchased a **model with multiple handsets**, the package should contain the following for each additional handset:

- Two batteries
- One battery cover
- One charging cradle
- One power adapter

The base station and charging cradle are designed for use in closed, dry rooms within a temperature range of +5 °C to +45 °C.

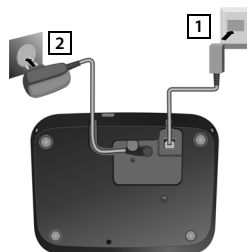
The device's feet do not usually leave any marks on surfaces. However, due to the multitude of different varnishes and polishes used on furniture, contact marks on the surfaces cannot be completely ruled out.

Never expose the telephone to heat sources, direct sunlight or other electrical devices.

Protect your telephone from moisture, dust, corrosive liquids and vapours.

Connecting the base

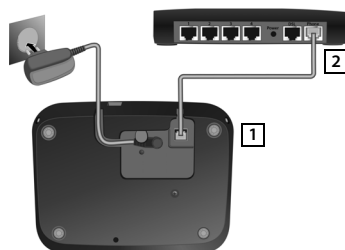
- ▶ **First** connect the phone jack for your country **1** and insert the cables in the cable ducts.
- ▶ **Then** connect the power adapter **2**.

Handset

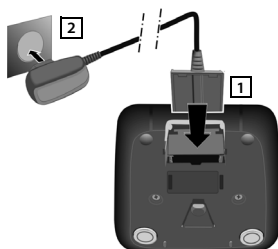
Alternatively: Connect the base station to a router

- ▶ Connect the phone socket of the device **1** and the phone socket of the router **2** with the phone cable supplied.

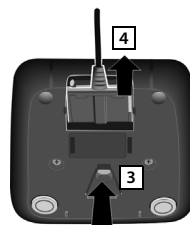
Handset

Connecting the charger (if included)

- ▶ Connect the flat plug on the power adapter **1**.
- ▶ Plug the power adapter into the plug socket **2**.

To remove the plug from the charging cradle again:

- ▶ Disconnect the power adapter from the mains power supply.
- ▶ Press the release button **3**.
- ▶ Pull out the plug **4**.

Setting up the handset for use

The display is protected by a plastic film. **Please remove the protective film!**

Inserting the battery and closing the battery cover

Only use a rechargeable battery, as this could otherwise result in significant health risks and personal injury. For example, the outer casing of the battery could be destroyed or the battery could explode. The device could also malfunction or be damaged as a result of using batteries that are not of the recommended type.

- ▶ Insert the batteries (for correct +/- direction, see diagram).

- ▶ Fit the cover from the top **1**.
- ▶ Then press the cover until it clicks into place **2**.

- To open the battery cover:
- ▶ Push the cover down **3**.

Charging the batteries

- ▶ Charge the batteries fully prior to first use in the charging cradle/base.

The batteries are fully charged when the power icon  disappears from the display.

Batteries may heat up during charging. This is not dangerous.

After a time, the charge capacity of the battery will decrease for technical reasons.

Handsets contained in the package have already been registered to the base. In the event that a handset is not registered however, (indicated by **Please register handset**), register the handset manually (→ p. 33).

Setting the language

Display language

When using the device for the first time, you are prompted in English for the language required for the display.

- ▶ ... Use  to select the language required ▶ **OK**

Handset

Setting the date and time

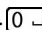
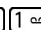
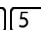
Set the date/time so that the correct date and time can be assigned to incoming calls, and so that you can use the alarm.

▶ Press the display key below the display **Time**. The entry field will open.

▶ use  to enter the day, month and year in 8-digit format,

e. g.         for April 1st 2019. ▶ **OK**

To correct errors: Press left or right on the control key  in order to swap between fields.

▶ Enter the hours and minutes in 4-digit format using the keypad, e.g.     for 07:15. Switch between the fields using the control key, if necessary. ▶ **OK**

The display shows **Saved**. You will hear a confirmation tone.

To revert to idle status: ▶ Press and **hold** the End call key .

Using the phone

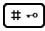
Getting to know your telephone

Activating/deactivating the handset

- ▶ Press and hold the end call key  (confirmation tone).

Activating/deactivating the keypad lock

The keypad lock prevents any inadvertent use of the phone.

- ▶ In idle status, press and **hold** the hash key  (confirmation tone). The keypad lock is activated or deactivated. If it is activated, the  icon appears in the display.

If the keypad lock is activated, a corresponding message is displayed when you press a key.

The keypad lock deactivates automatically when you receive a call. It is reactivated when the call is finished.

It is not possible to call emergency numbers either when keypad lock is activated. The SOS key still works even if the keypad lock is activated (→ p. 24).

Control Key

The control key enables you to navigate the menus and input fields, and also to call up certain functions depending on the situation.

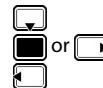
In the description below, the side of the control key (up, down, right, left) that you have to press in the different operating situations is marked in black, e.g.  for "press right on the control key" or  for "press the centre of the control key".

In idle status

Opening the directory

Opening the main menu

Opening the list of handsets

In submenus, selection and entry fields

Confirming a function

During a conversation

Opening the directory

Muting the microphone

Initiating an internal consultation call

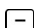
Getting to know your telephone

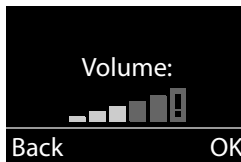
Volume keys

During a call, set the volume for the **receiver/headset** or **speaker** or in the case of an incoming call, set the volume of the **ringtone** (5 levels).

- ▶ Press the volume keys on the right side of the handset:

 (louder) /  (quieter)

The current setting is permanently saved after approx. two seconds (→ p. 36).

Extra-loud function

- ▶ Press the top volume key  until you reach the highest level.

The extra-loud setting only applies for the duration of the call.

- This setting can result in harm to health in persons with a normal hearing ability and is only intended for persons with a hearing impairment or reduced hearing.
- Disruptive noises on the phone line can be increased.

Direct dialling keys A to D

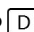
You can store particularly important numbers called frequently under the four **Direct dial keys**. You can assign an **external number** or an **internal call** to every direct dial key.

To dial these saved numbers, press the relevant direct dialling key  to .

You can also assign the emergency function to **direct dialling key**  (→ p. 24).

Assigning a function to direct dialling keys

Prerequisite: The direct dialling key in question has not already been assigned a function.

- ▶ In idle status, press the direct dialling key ( to ) on the handset to which you wish to assign a function.

Select the relevant function using .

- ▶ **Label with Ext. No.** Enter an external phone number directly.
 - ▶  Enter the number and press **OK**.
 - ▶  Enter the name and press **OK**.
- ▶ **Label with internal No.** Direct dial to a handset registered to the same base.
If more than one other handset is registered: ▶  Select handset ▶ **OK**
- ▶ **Label with Call All Int.** Direct dial to all handsets registered to the same base.
- ▶ **Label with Dir. Number** Label the key with a number from the directory. The directory is opened. ▶ Select a number and press **OK**.

Changing the function of the direct dialling keys

- ▶ **Menu** ▶  **Settings** ▶ **OK** ▶  **Direct Dial Keys** ▶ **OK** ▶  **Select direct dialling key (A to D)**

The current labelling is displayed after the key number, e.g.,

- ▶ **A: <no entry>** i.e., the  key is not currently labelled.

You can now change the labelling.

Select a new function using **Menu** ▶  Not all functions may be available, depending on the current assignment. An entry that is already present must be deleted in advance.

- ▶ **Delete Entry** Delete current key labelling.
- ▶ **Edit Entry** Change phone number and/or name.

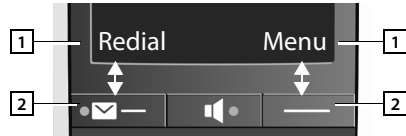
Only key : Assign key with emergency function → p. 24

Display keys

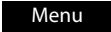
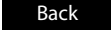
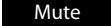
The functions of the display keys change depending on the particular operating situation.

Example:

- 1 Current display key function
- 2 Display keys

Examples:

	Open redial list.
	Open main menu/menu for further functions.
	Confirm selection or save entry.
	Missed calls or messages on the network mailbox (→ p. 21).
	Delete key: delete character by character/word by word from right to left.
	Go back one menu level or cancel operation.
	Mute the microphone (→ p. 18).

Display icons

	Action complete		Action failed
	Information		Alarm call (→ p. 29)
	Please wait ...		Display up/Down

Getting to know your telephone

Menu guidance

Your telephone's functions are accessed via a menu consisting of several levels.

Main menu (first menu level)

- ▶ When the handset is in idle status, press the display key **Menu** to open the main menu.

The functions in the main menu are shown by means of an icon and the function's name.

Selecting a function:

- ▶ You can scroll between functions using the control key . The selectable function is shown in the display.
- ▶ Press the display key **OK** to select the displayed function. This opens the corresponding submenu and the first entry is displayed.

If you press the display key **Back** or **briefly** press the end call key , the display returns to idle status.

Sub menu

The functions in the sub menu are indicated by name.

Selecting a function:

- ▶ You can scroll between functions using the control key . The relevant function is shown in the display.
- ▶ Press the display key **OK** to select the displayed function. This opens the corresponding submenu and the first entry is displayed.

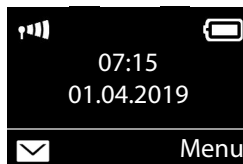
Briefly press the display key **Back** or the end call key  to return to the previous menu level.

Reverting to idle status

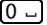
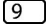
From any point in the menu:

- ▶ Press and **hold** the end call key  **or**
- ▶ Do not press any key: after 2 minutes, the display will **auto-**
matically revert to idle status.

Writing and editing names

The following rules apply when writing names:

- Each key between  and  is assigned several letters and characters.
- Control the cursor with  .
- Characters are inserted at the cursor position.
- Press the display key **Delete** to delete the character to the left of the cursor.
- The first letter of the name is automatically capitalised and then followed by lower case letters.

The characters assigned to the key are shown in a selection line at the bottom left of the screen. The selected character is highlighted.

- ▶ **Briefly** press the key several times in succession to select the required letter/character.

Setting upper/lower case or digits

Repeatedly press the hash key  to change the text input mode.

- 1 Writing digits
- A Upper case; First letter in capitals, all others in lower case
- a Lower case

The change of mode is displayed in the middle of the display.

Correcting incorrect entries

You can correct incorrect characters in the input fields by navigating to the incorrect entry using the control key . You can then:

- Use the display key **Delete** to delete the character to the left of the cursor,
- Insert characters at the cursor position
- Overwrite the highlighted (flashing) character e.g., when entering time and date.

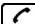
Making calls

Making calls

When the display backlight is switched off, pressing any key will activate the backlight.

Making an external call

External calls are calls to the public telephone network.

▶  Enter the number ▶ Press the Talk key  briefly

or

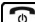
▶ Press and **hold** the Talk key ▶ ... use  to enter the number

Dialling the phone number:

- with the Direct dial key → p. 12
- from the directory → p. 22
- from the missed calls list → p. 20
- from the redial list → p. 19

Ending call/cancelling selection

▶ Press the End call key  or place the handset in the base or charging cradle.

Accepting a call

The handset indicates an incoming call in three ways: by ringing, by a display on the screen and by the flashing talk key . The signal light on the back of the handset flashes if the function **LED Call Signal** is activated (→ p. 37).

Options for answering a call:

- ▶ Press the Talk key .
- ▶ Press the Handsfree key .
- ▶ Press the display key **Accept**.

Switching the ringtone off:

- ▶ Press the display key **Silence**. You can accept the call while it is displayed on the screen.

If the handset is in the base and the **Auto Answer** function is activated (→ p. 36), the handset will accept a call automatically when you lift it out of the base.

Calling Line Identification

When you receive a call, the caller's number is displayed on the screen, if the following prerequisites are met.

- Your network provider supports CLIP, CLI.
 - CLI (Calling Line Identification): The caller's number is transmitted.
 - CLIP (Calling Line Identification Presentation): The caller's number is displayed.
- You have requested CLIP from your network provider.
- The caller has arranged CLI with the network provider.

Call display

When you receive a call, the phone number is shown in the display.

If the number of the caller is stored in your directory, the display is replaced by the corresponding directory entry.

The following is displayed in place of the number:

- **Extern. Call**, if no number is transmitted.
- **Withheld**, if the caller has withheld Calling Line Identification.
- **Unavailable**, if the caller has not arranged Calling Line Identification.

The ringer can be switched off for unknown calls (calls with Calling Line Identification withheld) (→ p. 32).

Handsfree operation

In handsfree mode, you activate the loudspeaker so that you can hear the caller without having to hold the handset to your ear. You therefore have both hands free and other people can listen in on the conversation.

Inform your caller before you use the handsfree function so that the caller knows a third party may be listening.

Activating/deactivating handsfree mode

Activating while dialling

Enter the number and press the handsfree key.

Switching between earpiece and handsfree mode

Press the handsfree key.

During a call activate or deactivate handsfree mode.

If you wish to place the handset in the charger during a call:

- ▶ Press and hold the handsfree key  while placing the handset in the charger and for a further 2 seconds. Otherwise, the call is cancelled.

Making calls

For instructions on adjusting the handsfree volume → p. 36.

Muting

You can deactivate your handset's microphone during a call.

Mute Press the display key to mute the handset.

On Press the display key to reactivate the microphone.

Press the control key (right) to activate/deactivate the microphone.

Changing the volume during a call

The volume is changed for the current mode (handsfree mode, receiver).

- ▶ Press the volume keys on the right-hand side of the handset: up : louder / down : quieter.

This setting is permanently saved (→ p. 36).

Extra-loud function: → p. 12

Making internal calls

Internal calls are free calls between handsets that you have registered to the same base.

- ▶ (briefly press). **Call All** is displayed. ▶ Select handset from the list (the actual handset is marked with a <) or **Call All** (collective choice). ▶

Holding down the key starts a call on **all** registered handsets immediately.

Accepting an internal call

Your telephone rings and the internal number of the caller (e.g., **INT 2**) or the assigned name appears in the display.

- ▶ Press the talk or handsfree key on the handset to accept the call.

Ending a call

- ▶ Press the end call key on the handset.

Changing the name of the handset: → p. 38

Transferring a call to another handset/internal consultation

External call, internal consultation:

- ▶ ▶ The list of handsets is opened. ▶ If more than two handsets are registered to the base, select one handset or **Call All**. ▶ or OK.

Transferring an external call: You have two options to transfer the call:

- ▶ Wait until the participant called answers, then hang up:
Press the End call key .

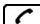
Or:

- ▶ Hang up before the participant answers: Press the End call key .

If the participant called does not answer or does not transfer the call, end the consultation call with **Back**.

Accepting a waiting call

If you receive an **external** call while conducting an **internal** call, you will hear the call waiting tone (short beep). The caller's number will appear in the display if phone number identification is enabled.

- ▶ Press the end call key  on the handset to end the internal call.
- ▶ Press the talk key  on the handset to accept the external call.

Lists

Using the redial list

The redial list contains the ten numbers last dialled with the handset (max. 30 digits). If one of the numbers is in the directory, the corresponding name will be displayed.

Dialling from the redial list

-  Press the Talk key.
-  Select an entry.
-  Press the talk key. The number is dialled.

Managing entries in the redial list

-  Press the Talk key.
-  Select an entry.
- Menu** Press the display key.

The following functions can be selected with :

- ▶ **Use Number**
- ▶ **Copy to Directory** Copy the caller's number to the directory.
- ▶ **Delete Entry** Delete selected entry.
- ▶ **Delete List** Delete all entries in the list.

Network mailbox list

You can use this list to listen to messages on the network mailbox (→ p. 27).

Lists

Missed calls list

Prerequisite: CLIP (→ p. 17)

Your telephone saves the last 25 missed calls.

Opening the list of missed calls

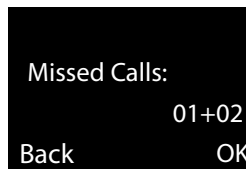
▶  ▶  Missed Calls

The list is displayed as follows:

Number of new messages + number of old, read messages.

OK Press the display key to open the list.

The last missed call is displayed as the first entry.

Example for a list entry:

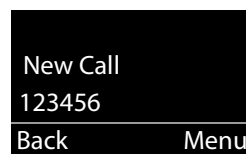
- Status of entry

New Call: New missed call.

Old Call: Entry already read.

- Number or name of caller

You can add the number of the caller to the directory (→ p. 23).

Menu functions

Use the **Menu** display key to select the following options:

- ▶ **Delete Entry** Delete selected entry.
- ▶ **Copy to Directory** Copy the caller's number to the directory.
- ▶ **Date and Time** Display date and time of the call.
- ▶ **Show Number** Display caller's phone number.
- ▶ **Delete List** Delete all entries in the list.

After exiting the list, all entries are assigned the status "old".

Selecting from the list of missed calls

Open list:  ▶  Missed Calls

 Select an entry.

 Press the talk key. The number is dialed.

Displaying CNIP information

If you have requested CNIP, you can view the name and town that are registered for this phone number with your network provider.

Select a list entry.

▶ **Menu** ▶ **Show Name**

If the name and town are not shown, it means that the caller has not requested Calling Line Identification or that Calling Line Identification has been withheld.

OK Press the display key to return to the list.

Deleting all entries

Caution! All **old and new** entries will be deleted.

Open list:  ▶  Missed Calls

▶ Menu ▶  Delete List ▶ OK

Calling up lists with new messages

Notifications about missed calls and messages on the network mailbox are saved in the messages list.

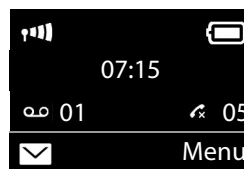
If there are **messages** in a list, use the display key  to call up:

- The network mailbox (if your network provider supports this function and the number of the network mailbox has been entered → p. 27)
- The missed calls list

As soon as a **new message** arrives at the answering machine, an advisory tone will sound. The Message key  also flashes.

Icons for message types and the number of new messages are shown on the idle display. The number of **new** entries is displayed beside the corresponding icon.

Icon	New message...
	... on the network mailbox
	... in the list of Missed Calls

Display messages

Answering machine

- ▶ Press the Message key  ▶  Answ.Mach. ▶ OK ▶  select the answering machine (Net AM) ▶ OK ...

The network mailbox number is dialled (further information → p. 27)

Missed calls

- ▶ Press the Message key  ▶  **Missed Calls** ... missed calls are listed (further information → p. 20)

New calls on the network mailbox are only displayed correctly if your network provider transmits this information (see user guide for your provider's network mailbox).

Directory

Directory

You can create a directory for your own individual handset. You can save numbers and relevant names in the directory.

Opening the directory

In idle status: ▶ Press the control key 

Length of an entry

Number: max. 30 digits

Name: max. 16 characters

Storing the first number in the directory

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Storing further numbers in the directory

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Refer to the character set chart to see how to enter names (→ p. 48).

Order of directory entries

The directory entries are sorted in alphabetical order. Spaces and digits take first priority. The sort order is as follows:

Space – Digits (0–9) – Letters (alphabetical) – Other characters

To get round the alphabetical order of the entries, insert a space or a digit in front of the name. These entries will then move to the beginning of the directory.

Selecting a directory entry

 Open the directory.

You have the following options:

- Use  to scroll through the entries until the required name is selected.
- Enter the first character of the name, if necessary scroll to the entry with the  key.

Dialling with the directory

▶  ▶  (Select entry) ▶ 

Managing directory entries

Select an entry.

Editing entries

- ▶ **Menu** ▶  **Edit Entry** ▶ **OK** ▶  (Change number if required) ▶ **OK** ▶  (Change name if required) ▶ **OK**

Using other functions

The following functions can be selected with **Menu** ▶ :

- ▶ **Show Number** Show number.
- ▶ **Delete Entry** Delete selected entry.
- ▶ **Send Entry** Send a single entry to another handset (→ p. 23).
- ▶ **Delete List** Delete all directory entries.
- ▶ **Send List** Send the complete list to another handset (→ p. 23).

Sending the directory to another handset

You can transfer all or individual entries to other handsets (→ p. 23). Transfers can also be made from old handsets, provided that these are registered to your (new) base.

You can transfer directory entries from other handsets to your new handset – even entries from old handsets.

The sending and receiving handsets must both be registered to the same base.
The other handset and the base can send and receive directory entries.

- ▶  ▶  (Select entry) ▶ **Menu** ▶  **Send Entry / Send List** ▶ **OK** ▶  (select the internal number of the receiving handset) ▶ **OK**

A successful transfer is confirmed by a message and confirmation tone on the receiving handset.

Adding a displayed number to the directory

You can copy numbers displayed in a list, e.g., the call list or redial list, to the directory.

A number is displayed.

- ▶ **Menu** ▶ **Copy to Directory** ▶ **OK**
- ▶ Complete the entry (→ p. 22).

Emergency function

Emergency function

Your handset is equipped with four additional keys (direct dial keys **A** to **D**). The **direct dial key A** can be assigned a special emergency function ("emergency key"). You can use this to request emergency assistance from friends or neighbours.

The emergency function must be set up first (→ p. 24)

Once the emergency function has been activated, the emergency key on the handset is permanently illuminated.

Process

▶ You can trigger the emergency call on the **handset** by pressing the direct dial key **A**.

The person receiving your emergency call will hear the emergency call message:

- "This is an emergency call."
- "To accept the call please press key 5."

If the person receiving the call presses the key 5, you can now speak to one another.

If the person receiving the call does not accept the call, **the emergency function automatically dials the next emergency number after 60 seconds** (if several numbers have been saved). The emergency function also automatically switches to the next emergency number after 60 seconds in the following situations:

- The answer machine of the emergency number selected is activated
- The emergency number is busy
- The telephone of the person receiving the emergency call is not set to "tone dialling".

This process is repeated a maximum of 5 times. If none of the calls are accepted, the emergency function is ended with an error tone.

The emergency key on the handset flashes throughout the entire process. The message "Sending emergency call" is shown in the display.

The telephone of the person receiving the emergency call must be set to tone dialling, otherwise the acceptance of the emergency call using the key 5 is not recognised.

Setting up the emergency function

In order to be able to use the function you must

- save the emergency numbers and
- activate the emergency function.

Emergency function

The police, ambulance service or fire service must not be used for an emergency call.

Initially setting up the emergency function

If you have already assigned the key **A** for the direct dial, you must first delete this number (→ p. 12). You can then save the emergency number on the direct dial key **A**.

▶ In idle status, press the direct dial key **A** on the handset.

▶ **Activate Emergency Function** ▶ **OK** (☑ = activated)

Entering a destination number: ▶ Enter the number ▶ **OK** ▶ Enter the name ▶ **OK**

You have the option of entering three additional numbers:

▶ **Add another number?**

Confirm by pressing **Yes**, if you would like to enter other emergency numbers and then enter the numbers. Confirm by pressing **No**, if you do not wish to add any further emergency numbers.

The display shows **Saved**.

The emergency function is now set up and is automatically activated.

Your emergency numbers:

1	
2	
3	
4	

- If your telephone is connected to a PABX, you must enter the access code (outside line code) as the first digit of your entry, if necessary (→ user guide of your PABX).
- You should carry out a test run to ensure that the emergency function has been correctly set up.

Changing the emergency numbers

▶ **Menu** ▶ **Settings** ▶ **OK** ▶ **Emergency Function** ▶ **OK** ▶ **Emergency Numbers** ▶ **OK** ▶ (Select entry 1 to 4) ▶ **Menu** ▶ **Edit Entry** ▶ **OK**

Enter the numbers and names as outlined above.

Emergency function

Deleting emergency numbers

- ▶ Menu ▶  Settings ▶ OK ▶  Emergency Function ▶ OK ▶  Emergency Numbers ▶ OK ▶  (Select entry 1 to 4) ▶ Menu ▶  Delete Entry ▶ OK

The entry has been deleted.

Activating/deactivating the emergency function

- ▶ Menu ▶  Settings ▶ OK ▶  Emergency Function ▶ OK ▶  Activation ▶ OK  = on) ... the key  is illuminated

Please note that if the emergency function is activated, the standby time of the handset will reduce.

If you have activated the **Eco Mode+** (→ p. 28), the wireless connection to the base is not displayed on the handset. The emergency key  remains illuminated, even if there is no longer a wireless connection.

You can check the contactability of the base by pressing **and holding** the End call key

. If the base is contactable, you will hear the dialling tone.

Triggering an emergency call

At least one emergency number has been saved (→ p. 25) and the emergency function is activated (the key  is illuminated) (→ p. 26).

- ▶ Press the direct dial key  on the handset, the emergency call commences.

An emergency call interrupts all active calls on the base.

If you have **accidentally triggered an emergency call**, end it by pressing the End call key .

The emergency key also works when the keypad lock is activated.

Using the network mailbox

The network mailbox is the answer machine in your provider's telephone network. More information is available from your provider.

You cannot use the network mailbox unless you have **requested** it from your provider.

Entering network mailbox number

In order to use the network mailbox quickly and easily via the network mailbox list and to use fast access, you will need to enter the number in your phone.

▶ **Menu** ▶ **Answer Machine** ▶ **OK** ▶ **Network Mailbox** ▶ **OK**

Enter the network mailbox number and press **OK**.

Accepting a new message from the network mailbox

When a message is recorded, you receive a call from the network mailbox. If you have requested Calling Line Identification, the display shows the network mailbox number. If you accept the call, the new messages are played back. If you do not accept the call, the network mailbox number is saved in the **missed calls list** (→ p. 20).

Enter the network mailbox phone number into your directory along with the designation "Net AM"; the display and the call list will then show this designation.

Playing back messages

There are three ways to call the network mailbox.

- Call the network mailbox via the **Answer Machine** menu:
 - ▶ **Menu** ▶ **Answer Machine** ▶ **OK** ▶ **Play Messages** ▶ **OK** ▶ **Network Mailbox** ▶ **OK**

- Access the network mailbox via the network mailbox list:

▶ ▶ **Answ.Mach.** (see also → p. 21)

The list is displayed as follows:

00: The number of new messages is only displayed correctly if your network provider transmits this information.

****:** The number of old messages is not available.

OK Press the display key to call up the network mailbox.

- Speed dialling the network mailbox: ▶ Press and **hold** the key .

Further functions

ECO DECT

You are helping to protect the environment with your Gigaset.

Reduced energy consumption

Your telephone has a power-saving adapter plug and uses <0.5 W when in idle status and when the batteries are not being charged.

Reducing radiation

The radiation from your telephone is reduced **automatically**:

- **Handset:** The closer the handset is to the base, the lower the radiation.
- **Base:** The radiation is reduced to virtually zero when only one handset is registered and the handset is placed in the base.

You can reduce the radiation from the handset and the base even more by using **Eco Mode**:

- **Eco Mode**
Reduces the radiation of the base and handset by 80% – whether you are making a call or not. **Eco Mode** reduces the range of the base by approx. 50%. Using **Eco Mode** always makes sense when a reduced range is sufficient.

Switching off radiation

- **Eco Mode+**
When you activate **Eco Mode+** the radiation (DECT transmission power) of base and handset is switched off automatically and only turns on when calls are made.
This is also true when multiple handsets are used, provided the handsets support **Eco Mode+**.

Eco Mode / **Eco Mode+** can be activated/deactivated independently of one another and can also be used with multiple handsets. The handset need not be placed in the base.

Activating/deactivating Eco Mode / Eco Mode+:

- ▶ Menu ▶  Settings ▶ OK ▶  System ▶ OK ▶  Eco Mode / Eco Mode+ ▶ OK
( = on).

Status displays

Display icon	Reception strength
	– Good to poor
 (red)	– No reception
	Eco Mode+ enabled (displays instead of the reception strength icon when in idle status)

Using the handset as an alarm clock

- When using **Eco Mode+** you can ensure that the range of the base is sufficient by **long** pressing the talk key . You hear the dial tone if the base can be reached.
- When **Eco Mode+** is enabled:
 - Call setup is delayed by approx. 2 seconds.
 - The standby time is reduced by about 50%.
- Registering handsets that do not support **Eco Mode+** causes the mode to be deactivated on the base and all other handsets.
- Activating **Eco Mode** reduces the range of the base.
- **Eco Mode / Eco Mode+** and repeater support (→ p. 39) cancel each other out i.e., if you use a repeater you cannot use **Eco Mode** or **Eco Mode+**.

Using the handset as an alarm clock

Activating/deactivating the alarm clock

The time is set (→ p. 10).

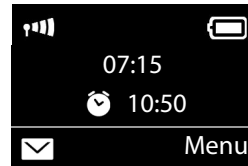
- ▶ Menu ▶ Alarm Clock ▶ OK ▶ Activation ▶ OK = on

After you activate the alarm clock, the menu for setting the wakeup time opens automatically.

Setting the wakeup time

- ▶ Menu ▶ Alarm Clock ▶ OK ▶ Wake-up Time ▶ OK ▶ Enter wakeup time ▶ OK

If the alarm clock is activated, you will see the icon in the display and the wakeup time after it.

The wakeup time is shown in the display only when all calls in the call lists have been read.

Setting the alarm clock signal

- ▶ Menu ▶ Alarm Clock ▶ OK ▶ Signal ▶ OK ▶ = on

Switching off/repeating the alarm after an interval (snooze mode)

Prerequisite: An alarm call sounds.

- ▶ **Deactivate until the next alarm call:** Press **Off**.

Or:

- ▶ **Snooze mode:**

Press **Snooze** or any key. The wake-up call is deactivated and then repeated after 5 minutes.

Or:

Press nothing. The alarm call is deactivated after 60 seconds and then repeated after 5 minutes. After the fourth repetition, the alarm call is deactivated for 24 hours.

Baby monitor

Baby monitor

When the baby monitor is switched on, the stored (internal or external) destination number is called as soon as a defined noise level is exceeded in the vicinity of the handset.

In baby monitor mode, incoming calls are only signalled on the display (**without ringtone**). The display backlight is reduced to 50%. Advisory tones are deactivated.

If you accept an incoming call, the baby monitor mode is suspended for the duration of the call, but the function **remains** activated. The baby monitor mode is not deactivated by switching the handset off and on again.

The handset should be positioned 1 to 2 m away from the baby. The microphone must point towards the baby.

Activating the function reduces the operating time of your handset. For that reason, place the handset in the charging cradle if necessary.

The baby monitor is activated 20 seconds after switching on.

The answer machine for the destination number must be switched off.

After switching on:

- ▶ Test sensitivity.
- ▶ Test the connection, if the alarm is being forwarded to an external number.

Activating and setting the baby monitor

- ▶ **Menu ▶ Baby Monitor ▶ OK ▶ OK** (☑ = baby monitor activated)

If a destination number has already been entered, the baby monitor is activated with this. The display shows **Baby Monitor :** and the specified destination number

If a destination number has not yet been entered:

Activate an **External call**: ▶ **OK** ▶  Enter the number ▶ **OK**

or activate an internal call:

- ▶  **Internal call** ▶ **OK** ... the baby monitor is triggered on all registered handsets

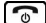
Changing the destination number

- ▶ **Menu ▶ Baby Monitor ▶ OK ▶**  **Call Target** ▶ **OK** ▶  select **External call** or **Internal call** ▶ **OK**

with **External call**: ▶  Insert destination number ▶ **OK**

Deactivating baby monitor/cancelling alarm

Deactivating the baby monitor: ▶ In idle status press the display key **Off**

Cancelling the baby monitor alarm: ▶ Press the End call key  during an alarm

One touch call

You can set up your phone so that you can dial a specific number when you press any key on the keyboard. This allows children, who are unable to enter a number, to call a certain number, for example.

▶ **Menu** ▶  **One Touch Clock** ▶ **OK** ▶  **Activation** ▶ **OK** ( = activated)

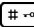
If you activate the one touch call, the menu automatically opens up afterwards in order to enter the destination number.

▶ **Call Target:** Enter or change the number that you wish to dial. ▶ **OK**

The activated One Touch Call appears on the idle display. The saved number is dialled by pressing any key. Press the End call key  to cancel dialling.

Ending one touch call mode

▶ Press display key **Off** ▶ Press and **hold** the  key.

or: ▶ Press and **hold** the  key

If you have already assigned the emergency function to the key , the emergency number is also selected if a one touch call is initiated.

Torch/Flashlight

You can use your telephone as a torch.

Activating/deactivating the torch function

Switch on: ▶ **Menu** ▶  **Flashlight** ▶ **OK** ... the torch is activated.

Switch off: ▶ **Off** (After 2 minutes the function is automatically deactivated.)

The function is automatically deactivated if the emergency function is pressed.

If the torch is activated, all functions are locked, however the emergency function still works.

The torch LED can also be used as an optical call signal for incoming calls (→ p. 37).

Protection against unwanted calls

Protection against unwanted calls

Black list

When the black list function is activated, calls from black list numbers are only signalled in the display. The setting only applies to handsets for which the setting is configured.

Editing the black list

▶ Menu ▶  Settings ▶ OK ▶  Audio / Visual Signals ▶ OK ▶  Ringtones (Handset) ▶ OK ▶ Black List ▶ OK ▶  Blocked Numbers ▶ OK

▶ New ▶ Enter number ▶ OK

Or:

▶ Delete ▶ Delete number ▶ OK

Activating the black list

▶ Menu ▶  Settings ▶ OK ▶  Audio / Visual Signals ▶ OK ▶  Ringtones (Handset) ▶ OK ▶ Black List ▶ OK ▶ Activation ▶ OK (= Function activated)

Time Control

You can specify a time period when you do not want the telephone to signal external calls e.g., during the night.

▶ Menu ▶  Settings ▶ OK ▶  Audio / Visual Signals ▶ OK ▶  Ringtones (Handset) ▶ OK ▶ Time Control ▶ OK

▶  Activation ▶ OK (= activated)

▶  Settings ▶ OK ▶ Starting Time: /Ending Time: Enter each time as a 4-digit number and press OK.

Anonymous Calls

You can set your handset so that it doesn't ring for calls where Calling Line Identification has been withheld (not answer machine). The call will only be signalled on the display.

▶ Menu ▶  Settings ▶ OK ▶  Audio / Visual Signals ▶ OK ▶  Ringtones (Handset) ▶ OK ▶ Anonymous Calls Silent ▶ OK (= activated)

Using multiple handsets

Registering handsets

You can register up to four handsets on the base.

Manual registering of Gigaset handsets

You must register the handset manually on both the handset (1) and the base (2).

After it has registered, the handset returns to idle status.

The registration process is cancelled after 60 seconds. If the handset has not registered within this time, repeat the process.

1) On the handset

- ▶ Menu ▶  Settings ▶ OK ▶  Registration ▶ OK ▶  Register Handset ▶ OK ▶ Enter the base system PIN (default setting: 0000 ▶ OK

2) On the base

- ▶ Within 60 seconds, press and **hold** the registration/paging key on the base for approx. 3 seconds.

Registering other handsets

You can register the handsets of other devices with GAP functionality as follows:

1) On the handset

- ▶ Start to register the handset as described in its user guide.

2) On the base

- ▶ Press and **hold** the registration/paging key on the base for approx. 3 seconds.

De-registering handsets

You can de-register any other registered handset from any registered Gigaset W570H handset.

- ▶ Menu ▶  Settings ▶ OK ▶  Registration ▶ OK ▶  De-register Handset ▶ OK

Select the internal party(ies) you wish to de-register and press **OK**.

(The handset you are currently using is indicated by <.)

Enter the current system PIN and press **OK**.

OK

Press the display key.

Using multiple handsets

Locating a handset ("Paging")

You can locate your handset using the base.

- ▶ **Briefly** press the registration/paging key on the base.
- ▶ All handsets will ring at the same time ("paging"), even if the ringers are switched off.

Ending paging

- ▶ **Briefly** press the registration/paging key on the base or press the talk key  on the handset.
-

Listening in to an external call

You want to listen in to an existing external call.

Prerequisite: The **Listening In** function must be activated.

Activating/deactivating the function:

- ▶ **Menu** ▶  **Settings** ▶ **OK** ▶  **Telephony** ▶ **OK** ▶  **Listening In** ▶ **OK** ( = on)

Listening in to the call:

- ▶ **Long** press of the talk key  on the handset. All participants hear a signal tone. **Conf. Call** is shown on the handset display.

Ending listening in:

- ▶ Press the end call key  on the handset; all participants hear a signal tone.

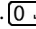
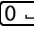
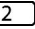
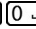
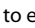
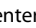
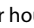
If the **first** internal participant presses the end call key , the handset that has "listened in" remains connected to the external participant.

Adjusting the telephone

Handset settings

You can change the settings on your handset to suit your individual requirements.

Setting the date and time

- ▶ **Menu** ▶  **Settings** ▶ **OK** ▶  **Date / Time** ▶ **OK**
- ▶ use  to enter the day, month and year in 8-digit format, e.g.         for April 1st 2019. ▶ **OK**
- ▶ use  to enter hours and minutes in 4-digit format, e.g.     for 07:15 ▶ **OK**

Changing the display language

You can view the display texts in different languages.

- ▶ **Menu** ▶  **Settings** ▶ **OK** ▶  **Language** ▶ **OK** ▶  (Select language) ▶ **OK**

The current language is indicated by .

If you accidentally choose a language you do not understand:

- ▶  ▶   ▶  (Select language) ▶ **OK**

Setting jumbo font

You can improve the legibility of the digits when you dial.

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When this function is deactivated, both the date and time are shown in idle status.

Changing the display lighting

Handset in the charging cradle:

- ▶ **Menu** ▶  **Settings** ▶ **OK** ▶  **Display** ▶ **OK** ▶  **Backlight** ▶ **OK** ▶ **In Charger** ▶ **OK** ( = on)

or off the charging cradle:

- ▶ **Menu** ▶  **Settings** ▶ **OK** ▶  **Display** ▶ **OK** ▶  **Backlight** ▶ **OK** ▶  ▶ **Out of Charger** ▶ **OK** ▶ **Standby time is reduced** ▶ **OK** ( = on).

Handset settings

Auto answer

When this function is activated, when a call arrives you can simply lift the handset out of the base without having to press the talk key .

▶ Menu ▶  Settings ▶ OK ▶  Telephony ▶ OK ▶  Auto Answer ▶ OK = on

Changing the call volume

You can set the handsfree volume and the earpiece volume to five different levels (1–5; e.g., volume 3 = .

▶ Menu ▶  Settings ▶ OK ▶  Audio / Visual Signals ▶ OK ▶  Call Volume ▶ OK ▶  Earpiece Volume / Handsfree Volume ▶ OK ▶  Set the volume ▶ OK

Setting the volume during a call: → p. 18

Changing ringers

- Volume: Five volume levels (1–5; e.g., volume 3 = .
- Melody: List of pre-loaded ringer melodies.

You can also set different ringer melodies for the following functions:

- External Calls
- Internal Calls

Setting the ringer volume

The ringer volume is the same for all types of ring.

▶ Menu ▶  Settings ▶ OK ▶  Audio / Visual Signals ▶ OK ▶  Ringtones (Handset) ▶ OK ▶ Volume ▶ OK ▶  Set the volume ▶ OK

Setting the ringer melody

Set different ringer melodies for external calls, internal calls.

For external calls you can also specify that your phone should not ring at certain times or if a call is from an unknown caller.

For internal calls:

▶ Menu ▶  Settings ▶ OK ▶  Audio / Visual Signals ▶ OK ▶  Ringtones (Handset) ▶ OK ▶ Melody ▶ OK ▶  Internal Calls ▶ OK ▶  (Select melody) ▶ OK = selected

For external calls:

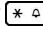
▶ Menu ▶  Settings ▶ OK ▶  Audio / Visual Signals ▶ OK ▶  Ringtones (Handset) ▶ OK ▶ Melody ▶ OK ▶  External Calls ▶ OK ▶  (Select melody) ▶ OK = selected

Activating/deactivating the ringer

You can:

- Permanently deactivate the ringer in **idle status** or **when receiving a call**
- Deactivate the ringer for the **current call** only.

The ringer cannot be re-activated while a call is in progress.

Deactivating the ringer permanently: ▶ Press and **hold** the star key ... the  icon appears in the display

Reactivating the ringer: ▶ Press and **hold** the star key .

Deactivating the ringer for the current call: ▶ Press the **Silence** display key.

Activating/deactivating visual call signal

Incoming calls are signalled visually (e.g. in noisy environments).

▶ Menu ▶  Settings ▶ OK ▶  Audio / Visual Signals ▶ OK ▶  LED Call Signal ▶ OK  = activated)

Advisory/warning tones

Your handset uses advisory tones to tell you about different activities and statuses. The following tones can be activated and deactivated independently of each other:

- **Advisory Tones**
 - **Key click:** Every key press is confirmed.
 - **Confirmation tone** (rising tone sequence): At the end of an entry/setting, when the handset is placed in the base and on arrival of a new entry in the call list.
 - **Error tone** (descending tone sequence): When you make an incorrect entry.
- **Battery Low Signal**
The battery requires charging.
- **Out of Range Signal**
The handset is no longer in range of the base.

You cannot deactivate the confirmation tone for placing the handset in the base.

Activating/deactivating advisory tones

▶ Menu ▶  Settings ▶ OK ▶  Audio / Visual Signals ▶ OK ▶  Advisory Tones ▶ OK  = on)

Activating/deactivating battery low beep

▶ Menu ▶  Settings ▶ OK ▶  Audio / Visual Signals ▶ OK ▶  Battery Low Signal ▶ OK  = on)

Handset settings

Activating/deactivating range warning tone

- ▶ Menu ▶  Settings ▶ OK ▶  Audio / Visual Signals ▶ OK ▶  Out of Range Signal ▶ OK (on)
-

Changing the name of the handset

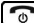
The base automatically specifies a name for the handset. You can change this name.

- ▶  ... the list of registered handsets is displayed ▶  Select handset ▶ Name ▶  Enter name ▶ OK
-

Restoring the handset default settings

You can reset any individual settings and changes that you have made. Entries from the directory and the call list, date and time, emergency numbers and registration of the handset on the base are retained.

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Cancel the reset with .

Base settings

The base settings are made using a registered handset.

Activating/deactivating music on hold

▶ Menu ▶  Settings ▶ OK ▶  Telephony ▶ OK ▶  Music on Hold ▶ OK = on

Activating repeater support

You can increase the range and signal strength of the base using a repeater. You will need to activate repeater mode. This will terminate any calls being made via the base at the time.

Prerequisite: A repeater is registered.

▶ Menu ▶  Settings ▶ OK ▶  System ▶ OK ▶  Repeater Mode ▶ OK = activated

After activating or deactivating the repeater, switch your handset off and on again (→ p. 11).

- **Eco Mode / Eco Mode+** (→ p. 28) and repeater support cancel each other out i.e., if you use a repeater, you cannot use **Eco Mode** or **Eco Mode+**.
- The default encrypted transmission setting is deactivated when repeater mode is activated.

Protecting against unauthorised access

Protect the system settings of the base with a system PIN known only to yourself. You will need to enter the system PIN when, for example, registering/de-registering a handset to/from the base.

Changing the system PIN

You can change the base's 4-digit default system PIN ("0000") to a 4-digit PIN known only to yourself.

▶ Menu ▶  Settings ▶ OK ▶  System ▶ OK ▶  System PIN ▶ OK ▶  System PIN: (enter the current system PIN) ▶ OK ▶ New PIN: (enter your new system PIN) ▶ OK

Resetting the system PIN

If you have forgotten your system PIN, you can reset the base to the original code **0000**:

Disconnect the power cable from the base. Hold down the registration/paging key on the base while reconnecting the power cable to the base. Hold down the key for at least 5 seconds.

The base has now been reset and the system PIN set to **0000**.

- All handsets are de-registered and must be re-registered. All settings are reset to the default settings.

Appendix

Customer Service & Assistance

Do you have any questions? As a Gigaset customer, you can find help quickly in this User Manual and in the service section of our Gigaset online portal www.gigaset.com/service.

In our constantly updated online service you can find:

- Extensive information about our products
- FAQ compilations
- Keyword search to help find topics quickly
- Compatibility database: Find out which base stations and handsets can be combined.
- Product comparison: Compare the features of several products with each other.
- Downloads for user manuals and current software updates
- E-mail contact form for customer service

In order to contact our customer service via email, please use the email contact form from our Gigaset online portal after selecting your home country.

Our representatives are also available on the telephone hotlines for consultation.

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Please have your record of purchase ready when calling. After sales service is not offered in countries where our product is not sold by authorised dealers.

Questions and answers

Possible solutions are available online at www.gigaset.com/service

In addition, the following overview provides troubleshooting assistance.

The display is blank

- The handset is not switched on: ► Press and **hold** the End call key .
- The battery is empty: ► Charge the battery or replace it (→ p. 9)

You can only hear the person you are speaking to in one ear

- Move the handset nearer to the hearing system

The device will not charge

- The handset is not inserted correctly in the charging cradle: ► Insert handset correctly

The device turns itself off even though the battery is still 2/3 charged according to the display

- The contacts/batteries are unclean or corroded ► Clean the contacts/batteries or replace the batteries (→ p. 9)

Call is interrupted or completely disconnected

- Radio signals are weakened by walls, ceilings, insulation etc. ► Do not place the base behind/beneath metallic objects. Do not place the base in the cellar if possible (→ p. 7)

Handsfree sound quality is not good

- Dirt on the handset (e.g. make-up, dust, iron filings) ► Clean the handset (→ p. 45)

Loud crackling, background noises, interference, scratching, changing volume

- Interference from other devices (network components of fax, PC, printer, mobile phone) ► Increase the distance to other devices (at least half a metre) (→ p. 7)

You cannot make calls and "BASE" or "No base" is flashing in the display

- The power cable is not connected to the base: ► Check the base power adapter (→ p. 7)

"Not possible!" appears in the display

- Telephoning externally is not possible whilst another external call is being made. ► Wait until the other call has ended (does not apply if the internal listening function has been activated).
- Making an external call whilst another party is speaking on your answer machine. ► Wait until the announcement on the answer machine has ended

Time is reset to the default setting

- Base and handset were deactivated at the same time ► Reset the time (→ p. 10)

No time is specified in the call list

- Date/time is not set ► Set date/time (→ p. 10)

Consultation call, call swapping, conference not possible.

- These functions that can be executed on the device must be provided and enabled by the network provider. Operation can vary from one network provider to another. ► Contact your network provider

Volume of the handset (earpiece volume) too quiet

- The volume is at the lowest level
 - Change the earpiece volume setting (→ p. 36)
 - Change the handset position on the ear

Handsfree volume too quiet

- The volume is at the lowest level ► Change the handsfree volume setting (→ p. 36)

You hear a tone during operation

- Action has failed/invalid input ► Repeat the process. Read the display and refer to the user guide
- The batteries are empty ► Charge the batteries (→ p. 9)
- The system PIN you have entered is incorrect ► Re-enter the system PIN or reset (→ p. 39)

You hear a tone during the call

- The batteries are empty ► Charge the batteries (→ p. 9)

Forgotten system PIN

- Reset the system PIN to 0000 (→ p. 39)

Error tone sounds after system PIN prompt

- The system PIN you have entered is incorrect ► Re-enter the system PIN or reset (→ p. 39)

The device has come into contact with liquid

- Allow the device to dry. ► Contact with liquid (→ p. 46)

The emergency function could not be configured

- You have not saved any emergency numbers ► Save at least one emergency number (→ p. 24)
- The emergency function is not activated ► Activate the emergency function (→ p. 24)
- The emergency function only works when the device is directly connected to the network or to a PABX that does not require "R" or "P" for network assignment ► Emergency function direct dial key (→ p. 24)

Manufacturer's advice

The receiver of an emergency call does not pick up

- In some PABX, acceptance of the emergency call by pressing the key 5 does not divert this to the connected handset. The emergency function is not possible in this case, as a conversation cannot be held.
 - ▶ Test the emergency function after set-up with each receiver entered.

Dealer or local after-sales services (Southern Africa)

In the event of an after-sales query/issue/claim please refer back to your point of purchase.

Proof of purchase (receipt) has to be submitted.

Exclusion of liability

Your handset's display consists of pixels. Each pixel consists of three sub-pixels (red, green, blue).

It may be the case that a sub-pixel is missing or has a colour deviation.

A warranty claim is only valid if the maximum number of permitted pixel errors is exceeded.

Description	Maximum number of permitted pixel errors
Colour illuminated sub-pixels	1
Dark sub-pixels	1
Total number of coloured and dark sub-pixels	1

Signs of wear on the display and casing are excluded from the warranty.

The Gigaset W570 is a premium large button cordless phone that relays phone calls directly to the hearing systems of Widex (Widex A/S, Nymoellevvej 6, 3540 Lyngby, Denmark).

No liability is assumed for the hearing systems or for any damage resulting from using the hearing systems. If you have any questions, please contact your specialist hearing systems retailer.

Manufacturer's advice

Authorisation

This device is intended for connection to analogue networks outside the European Economic Area (with the exception of Switzerland) depending on national type approval.

Country-specific requirements have been taken into consideration.

Hereby, Gigaset Communications GmbH declares that the radio equipment type

Gigaset W570 - Gigaset W570H is in compliance with Directive 2014/53/EU.

The full text of the EU declaration of conformity is available at the following internet address:

www.gigaset.com/docs.

This declaration could also be available in the "International Declarations of Conformity" or "European Declarations of Conformity" files.

Therefore please check all of these files.

Data protection

We at Gigaset take the protection of our customers' data very seriously. It is precisely for this reason that we are ensuring all our products feature "Privacy by Design" as standard. All information we collect is used to make our products as good as possible. In the process, we ensure your details are protected and only used for the purposes of making available to you a product or service. We know which path your data takes through the company and ensure this happens in line with data protection specifications in a secure and protected manner.

The full text of the privacy policy is available from: www.gigaset.com/privacy-policy

Environment

Our environmental statement

We at Gigaset Communications GmbH are aware of our social responsibility. That is why we actively take steps to create a better world. In all areas of our business – from product planning and production to sales and waste of disposal – following our environmental conscience in everything we do is of utmost importance to us.

Learn more about our earth-friendly products and processes online at www.gigaset.com.

Environmental management system

Gigaset Communications GmbH is certified pursuant to the international standards ISO 14001 and ISO 9001.

ISO 14001 (Environment): Certified since September 2007 by TÜV SÜD Management Service GmbH.

ISO 9001 (Quality): Certified since 17/02/1994 by TÜV SÜD Management Service GmbH.

Disposal

Batteries should not be disposed of in general household waste. Observe the local waste disposal regulations, details of which can be obtained from your local authority.

All electrical and electronic products should be disposed of separately from the municipal waste stream via designated collection facilities appointed by the government or the local authorities.

This crossed-out wheeled bin symbol on the product means the product is covered by the European Directive 2012/19/EU.

The correct disposal and separate collection of your old appliance will help prevent potential negative consequences for the environment and human health. It is a precondition for reuse and recycling of used electrical and electronic equipment.

For more detailed information about disposal of your old appliance, please contact your local council refuse centre or the original supplier of the product.

Care

Wipe the device with a **damp** cloth or an antistatic cloth. Do not use solvents or microfibre cloths.

Never use a dry cloth; this can cause static.

In rare cases, contact with chemical substances can cause changes to the device's exterior. Due to the wide variety of chemical products available on the market, it was not possible to test all substances.

Impairments in high-gloss finishes can be carefully removed using display polishes for mobile phones.

Manufacturer's advice

Contact with liquid

If the device comes into contact with liquid:

- 1 **Disconnect the power supply.**
 - 2 **Remove the batteries and leave the battery compartment open.**
 - 3 Allow the liquid to drain from the device.
 - 4 Pat all parts dry.
 - 5 Place the device in a dry, warm place **for at least 72 hours** (not in a microwave, oven etc.) with the battery compartment open and the keypad facing down (if applicable).
 - 6 **Do not switch on the device again until it is completely dry.**
- When it has fully dried out, you will normally be able to use it again.

Specifications

Batteries

Technology:	Nickel-metal-hydride (NiMH)
Size:	AAA (Micro, HR03)
Voltage:	1.2V
Capacity:	750 mAh

The device is supplied with two approved batteries.

Handset operating times/charging times

The operating time of the telephone depends on the capacity of the battery, its age and the way it is used. (All times are maximum possible times.)

Standby time (hours) *	320 * / 200 **
Talktime (hours)	14
Operating time for 1.5 hours of calls per day (hours) *	130 * / 100 **
Charging time in base (hours)	9
Charging time in charging cradle (hours)	7.5

***Eco Mode** deactivated, **without** display backlight in idle status

****Eco Mode** activated, **without** display backlight in idle status

Base power consumption

In standby mode

- Handset in charging cradle approx. 0.70 W
- Handset away from charging cradle approx. 0.50 W

During a call approx. 0.70 W

DECT specifications

DECT standard	Supported
GAP standard	Supported
No. of channels	60 duplex channels
Radio frequency range	1880–1900 MHz
Duplex method	Time multiplex, 10 ms frame length
Pulse repetition rate	100 Hz
Pulse transmission length	370 µs
Channel grid	1728 kHz
Bit rate	1152 kbit/s
Modulation	GFSK
Language code	32 kbit/s
Transmission power	10 mW average power per channel, 250 mW pulse power

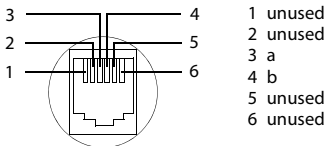
Specifications

Range	Up to 50 m indoors, up to 300 m outdoors
Base power supply	230 V ~/50 Hz
Environmental conditions for operation	+5 °C to +45 °C; 20% to 75% relative humidity
Dialling mode	DTMF (tone dialling)

WIDEX specifications

	WidexLink™
Frequency	10.6 MHz
Deviation	± 200 kHz
RF bandwidth	660 kHz (-15 dB)
Channels	Single channel
Modulation	FSK
Radiated avg. output power	13 nW (~ -48.8 dBm)
Magnetic field strength	-15 dBµA/m @ 10 m
Duty cycle	100% simplex
Audio delay	< 10ms

Pin connections on the telephone jack

Characters

Standard characters

	1x	2x	3x	4x	5x	6x	7x	8x	9x	10x	11x	12x	13x	14x	15x
1	1	£	\$	¥	α										
2	a	b	c	2	ä	á	à	â	ã	ç					
3	d	e	f	3	ë	é	è	ê							
4	g	h	i	4	ï	í	ì	î							
5	j	k	l	5											
6	m	n	o	6	ö	ñ	ó	ò	ô	õ					
7	p	q	r	s	7	ß									
8	t	u	v	8	ü	ú	ù	û							
9	w	x	y	z	9	ÿ	ý	æ	ø	ã					
0	←	.	,	?	!	0	+	-	:	¿	¡	"	'	;	_
*	*	/	()	<	=	>	%							
#	#	@	\	&	§										

1) Space

Index

A

Access protection	39
Activating	
advisory tones	37
auto answer	36
handset	11
keypad lock	11
Activating/deactivating	
emergency function	26
Adapter plug	
power saving	28
Advisory tones	37
Alarm	
baby monitor	30
Alarm clock	29
Anonymous Calls	32
Anonymous calls off	32
Authorisation	44
Auto answer	16, 36

B

Baby monitor	30
Base	
changing system PIN	39
connecting	7
operating on PABX	40
restoring default settings	40
setting	39
system PIN	39
Battery	
battery low tone	37
charging	9
inserting	9
Battery low tone	37
Black list	32
Boost key	2
Broken display	6

C

Call	
accepting	16
internal	18
participant listening in	34
transferring (connecting)	18
Call list	19
Call lists	20
Call protection	32
Call waiting, internal call	19
Calling	
internal	18
Calling back a caller	20
Calling back from a call list	20
Calling Line Identification	17
Calls	
making an external call	16

transferring (connecting)	18
Care	45
Care of the device	45
Changing	
display language	9
system PIN	39
Changing the name of the handset	38
CLI, CLIP	17
CNIP	20
Confirmation tone	37
Contact with liquid	46
Contents of the package	7
Control key	2
Correcting incorrect entries	15
Customer Care	41
Customer Service & Assistance	41

D

Data protection	45
Deactivating	
advisory tones	37
auto answer	36
handset	11
keypad lock	11
Dealer or local after-sales services	44
Delete key	13
Deleting characters	15
De-registering (handset)	33
Dialling	
directory	22
Dialling mode	40
Direct dial	
on PABX	25
saving numbers	12
Direct dialling	
saving/changing key assignment	12
Direct dialling key A	2
Direct dialling keys B to D	2
Directory	22
order of entries	22
selecting entry	22
sending entry/list to handset	23
storing entry	22
storing the first number	22
Display	
broken	6
changing display language	35
changing the display language	9
in idle status	14
network mailbox message	27
number (CLI/CLIP)	17
setting	35
Display keys	2, 13
Disposal	45
Do not disturb list, see Black list	

Index

E

Earpiece mode	17
Earpiece volume	36
ECO DECT	28
Eco Mode	28
Eco Mode+	28
Emergency numbers	
not possible	6
End call key	2, 16
Entering recall	2
Environment	45
Error tone	37
Exclusion of liability	44
Extra-loud function	12

G

General troubleshooting	42
-------------------------	----

H

Handset

activating/deactivating	11
advisory tones	37
changing the name	38
de-registering	33
display language	9, 35
handsfree volume	36
idle status	14
jumbo font	35
muting	18
registering	33
restoring default settings	38
searching for	34
setting (individual)	35
setting up	9
transferring a call	18
using multiple	33
Handsfree key	2
Handsfree operation	17
Handsfree volume	36
Hash key	2
Hearing aids	6
Help	42

I

Icon

keypad lock	11
ringer	37

Icons

displaying new messages	21
-------------------------	----

Idle status

returning to	14
--------------	----

Idle status (display)	14
-----------------------	----

Incorrect entries (correction)	15
--------------------------------	----

Internal

consultation call	18
-------------------	----

Internal call	18
---------------	----

call waiting	19
--------------	----

Internal consultation call	18
----------------------------	----

Internal listening in	34
-----------------------	----

J

Jumbo font	35
------------	----

K

Key 1 (fast access to network mailbox)	2
--	---

Key click	37
-----------	----

Keypad lock	11
-------------	----

Keys

boost key	2
-----------	---

control key	2
-------------	---

delete key	13
------------	----

display keys	2, 13
--------------	-------

end call key	2
--------------	---

fast access to network mailbox	2
--------------------------------	---

handsfree key	2
---------------	---

hash key	2
----------	---

key 1 (speed dialling the network mailbox)	27
--	----

message key	21
-------------	----

on/off key	2
------------	---

paging key	3
------------	---

recall key	2
------------	---

speed dialling the network mailbox	27
------------------------------------	----

star key	2
----------	---

talk key	2
----------	---

volume key	2
------------	---

L

Liquid	46
--------	----

List

call list	19
-----------	----

redial list	19
-------------	----

Listening in to a call	34
------------------------	----

Lists

call list	20
-----------	----

missed calls	20
--------------	----

Lock

activating/deactivating keypad lock	11
-------------------------------------	----

M

Making an external call	16
-------------------------	----

Making calls

accepting a call	16
------------------	----

internal	18
----------	----

Medical equipment	6
-------------------	---

Menu

using	14
-------	----

Message

playing back	27
--------------	----

Message key	2
-------------	---

opening lists	21
---------------	----

Missed call	20
-------------	----

Music on hold	39
---------------	----

Muting the handset	18
--------------------	----

N

Network mailbox	27
entering number	27
Number	
displaying caller's number (CLIP)	17
entering netw. mailbox number	27
storing in directory	22, 23

O

On/Off key	2
One touch call	31
Order of entries in directory	22

P

PABX	
operating base on PABX	40
setting dialling mode	40
setting recall	40
switching to tone dialling	40
Paging	3, 34
PIN change	
system PIN	39
Pin connections	48
Playing back	
message (answer machine)	27
Power adapter	6
Power consumption	47
Power saving adapter plug	28
Protecting phone against access	39
Protection from calls	32

Q

Questions and answers	42
Quick dialling, refer to Direct dialling	

R

Radiation	
reducing	28
Range warning tone	38
Recall key	2
Reception booster, see Repeater	
Redial	19
Reduced energy consumption	28
Reducing radiation	28
Registering (handset)	33
Repeater	39
Restoring default settings	
base	40
handset	38
Ring delay	36
Ringer	
changing	36
setting melody	36
setting volume	36
Ringtone	
silencing for anonymous calls	32

S

Safety precautions	6
Searching	
handset	34
in directory	22
Sending	
directory entry to handset	23
Setting melody (ringer)	36
Setting recall	40
Setting the date	10
Setting the time	10
Setting the volume	12
Setting the wakeup time	29
Setting up	
handset	9
Setting up the emergency call	24
Setting up the emergency function	24
Setting volume	
earpiece	36
handset handsfree	36
loudspeaker	36
ringer	36
Show new messages	21
Snooze mode	29
Specifications	47
Speed dial	
network mailbox	27
Standard characters	48
Star key	2
System settings	39

T

Talk key	2
Telephone jack	
pin connections	48
Time Control	32
Tone dialling	40
Torch	31

U

Unknown	17
Unknown caller	17

V

Viewing	
message	27
network mailbox	27
Volume	
ringer	36
Volume key	2
Volume keys	12

Index

W

Warranty 44

Widex

 cancelling phone coupling 1

 changing volume 1

 coupling phone and hearing system 1

 Widex hearing systems 1

 Writing and editing text 15

All rights reserved. Rights of modification reserved.

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