 **WSAUD A/S** Nymoellevej 6, DK-3540 Lyngby, Denmark
<https://global.widex.com>

 **Manual no.:**
9 514 0313 082 #05
Issue:
2020-11

9 514 031 3082 08





COM-DEX with firmware 1.3 and newer

DEX™
A WIRELESS WORLD

COM-DEX

GB
CZ
SK
HU
HR
CS
SI
BG
GR
RO
BO
AL
UA

WIDEX®
HIGH DEFINITION HEARING

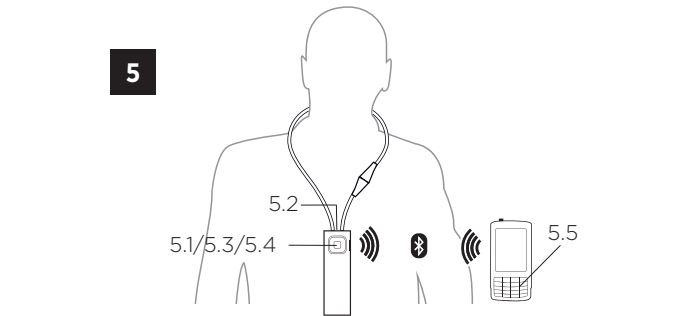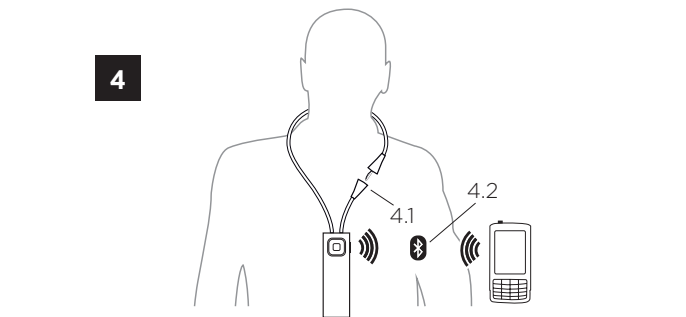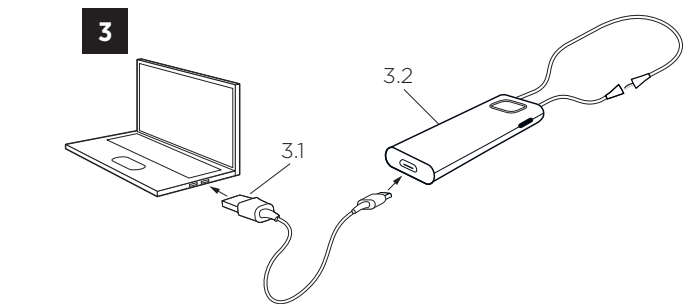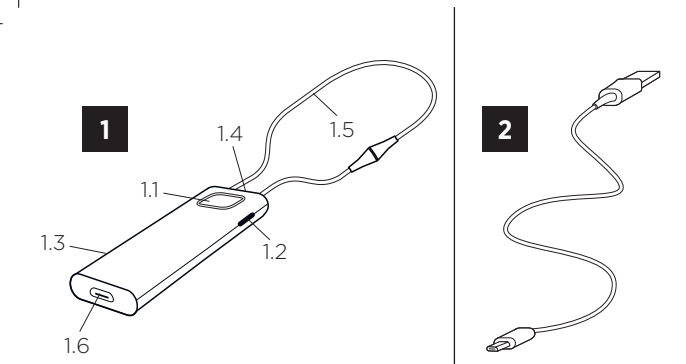GB	USER INSTRUCTIONS	2
CZ	NÁVOD K POUŽITÍ	14
SK	NÁVOD NA POUŽITIE	26
HU	HASZNÁLATI UTASÍTÁS	38
HR	KORISNIČKE UPUTE	50
CS	UPUTSTVO ZA UPOTREBU	62
SI	NAVODILA ZA UPORABO	74
BG	УПЪТВАНЕ ЗА УПОТРЕБА	86
GR	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	99
RO	INSTRUCTIUNI E DTLIUZARE	112
BO	KORISNIČKO PUTUSTVO	125
AL	UDHËZIMET E PËRDORIMIT	137
UA	ІНСТРУКЦІЯ З ЕКСПЛУАТАЦІЇ	150
	FCC AND ISED STATEMENTS	163

DESCRIPTION OF DEVICE – *See illustration 1*

- 1.1 Push button
- 1.2 Room Off button
- 1.3 LED
- 1.4 Microphone
- 1.5 Neck loop
- 1.6 Micro USB

Note: Read these instructions carefully before you start using the device.

Intended use

The device is intended to improve the ability of hearing impaired people to conduct mobile phone conversations while using their hearing aids, to improve the usability and ease of control of such hearing aids by equipping them with a Smartphone-based control method.

Note: The device uses Bluetooth standard which is common in newer mobile phones.

ACCESSORIES – *See illustration 2*

2.1 USB cable

CHARGING – *See illustration 3*

- 3.1 Connect USB cable between the device and the power supply. We recommend that the two neck loop parts are not joined while charging the device.
- 3.2 Red light: charging, green light: fully charged.

Battery life: 8 hours streaming, 8 days standby.

HOW TO USE – *See illustration 4*

- 4.1 Join the two neck loop parts. The neck loop functions as an on/off switch.
- 4.2 Pair the device to your mobile phone using Bluetooth.

How to pair the device with your phone

To pair your device with your phone for the first time, follow these steps:

1. Make sure that the device and phone are within 1 metre range of each other.
2. Connect the neck loop. The device will blink red and blue.
3. Activate Bluetooth on your phone.
4. Start the pairing process according to the instructions in your phone's user guide.
5. Your phone will show a list of Bluetooth devices.
Select COM-DEX from the list.

Once you have paired the device, it will remain so even when you disconnect the neck loop.

Incoming calls – *See illustration 5*

- 5.1 Answer call: Press push button.
- 5.2 Speak into microphone while device is hanging around your neck.
- 5.3 End call: Press push button.
- 5.4 Reject call: Long press on push button.

Note: Answering and ending calls is also possible on the mobile phone. Note that, on some mobile phones, you may need to select the COM-DEX from a list of devices. Otherwise the sound will be transmitted through the phone's speakers.

Outgoing calls – *See illustration 5*

- 5.5 Dial from phone.

Note: You can also use the device to stream sound from any sound source to your hearing aids.

Sound adjustment during use

Volume is adjusted on the mobile phone.

To shut out surrounding sounds: press Room Off button – See *illustration 1*.

Note: If you are in a particularly noisy environment or if you need to increase the volume to a higher level than normal while streaming sound to your hearing aids, keep the Room Off button pressed for 5 seconds. To return to your normal sound level, press the Room Off button again and keep it pressed for 5 seconds.

You can download the COM-DEX app from the App Store or Google Play. This app allows you to use your phone as a remote control.

MAINTENANCE

- Clean with a damp cloth. Do not use chemicals.
- Do not expose to extreme temperatures or high humidity.
- Do not immerse in liquids.
- When the device is not in use keep it in a dry location out of reach of children and pets.
- Never try to open or repair the device yourself. This may only be done by authorised personnel.

INFORMATION ABOUT EMISSION

Emission levels from Widex devices are several times lower than the levels transmitted by mobile phones and other communications equipment.

For further information, especially regarding patients with medical implants, download the document: “Information on Widex-Link™” at <https://global.widex.com/widexlink>

TROUBLESHOOTING

Problem	Possible cause	Solution
The device does not work	a) Device battery low b) The mobile phone is set to silent mode c) Hearing aid battery low d) Out of range	a) Charge device battery b) Set to normal mode c) Change hearing aid's battery d) Bluetooth has a range up to 10 m.
No sound	a) Not connected via Bluetooth	a) Repeat the pairing process

If the problems persist, contact your hearing care professional.

WARNINGS

- Do not leave the device unattended while charging.
- Your device should be stored and transported within the temperature and humidity ranges of -10° to +50°C (14°F to 122°F) and <90% RH.
- Your device is designed to operate from 5°C (41°F) to 40°C (104°F).
- Keep the device protected from dust or external particles.
- We recommend using a 5V up to 500 mA USB adaptor.
- Do not use the device in mines or other areas with explosive gases.
- When the device is connected via micro USB cable to external mains-operated equipment such as a charger, PC, audio source or similar, this equipment must comply with IEC 60950-1, IEC 62368-1, IEC 60601-1 or equivalent safety standards.
- Only the accessories and cable that are included in the sales box must be used.
- Using cables other than those specified or provided may result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

CONDITIONS FOR USE

Operate the device within the following conditions:

	Minimum	Maximum
Temperature	5°C (41°F)	40°C (104°F)
Humidity	10% rH	90% rH

Store and transport the device within the following conditions:

	Minimum	Maximum
Temperature	-10° (14°F)	50°C (122°F)
Humidity	10% rH	90% rH
Atmospheric pressure	750 mbar	1060 mbar

You can find technical datasheets and additional information on your device on <https://global.widex.com/doc>

REGULATORY INFORMATION

Directive 2014/53/EU

WSAUD A/S hereby declares that this COM-DEX is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU.

The device contains radio transmitters operating at: 10.6 MHz, -54 dB μ A/m @10 m, 2.4 GHz, 0.9 mW EIRP.

A copy of the Declaration of Conformity can be found at:
<https://global.widex.com/doc>

SYMBOLS

Symbols commonly used by WSAUD A/S in medical device labelling
(labels/IFU/etc.)

Symbol	Title/Description
	Manufacturer The product is produced by the manufacturer whose name and address are stated next to the symbol. If appropriate, the date of manufacture may also be stated.
	Date of manufacture The date when the product was manufactured.
	Batch code The product's batch code (lot or batch identification).
	Catalogue number The product's catalogue (item) number.
	Serial number The product's serial number.*
	Consult instructions for use The user instructions contain important cautionary information (warnings/precautions) and must be read before using the product.

Symbol	Title/Description
	Caution/Warning Text marked with a caution/warning symbol must be read before using the product.
	WEEE mark “Not for general waste” When the product is to be discarded, it must be sent to a designated collection point for recycling and recovery.
	CE mark The product is in conformity with the requirements set out in European CE marking directives.
	RCM mark The product complies with electrical safety, EMC and radio spectrum regulatory requirements for products supplied to the Australian or New Zealand market.
	Interference Electromagnetic interference may occur in the vicinity of the product.

*The six- or seven-digit number on the product is the serial number. Serial numbers may not always be preceded by **SN**

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- 1.1 Tlačidlo
- 1.2 Tlačidlo pre potlačenie okolitých zvukov
- 1.3 Dióda LED
- 1.4 Mikrofón
- 1.5 Závesná slučka
- 1.6 Port mikro USB

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FCC ID: TTY-CMDEX

IC: 5676B-CMDEX

Federal Communications Commission Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this

equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Changes or modifications to the equipment not expressly approved by WSAUD A/S could void the user's authority to operate the equipment.

ISED STATEMENT:

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

1. This device may not cause interference.
2. This device must accept any interference, including interference that may cause undesired operation of the device.

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ISED RADIATION EXPOSURE STATEMENT:

This equipment complies with ISED RF radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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