
**TV PLAY™**  
STAY CONNECTED

USER INSTRUCTIONS  
**TV PLAY**

SK  
HU  
HR  
RS  
SI  
BG  
GR  
LT  
RU  
RO  
EE  
BA  
LV  
UA  
AL  
TR  
SA  
IR  
IL  
PH  
PK  
GB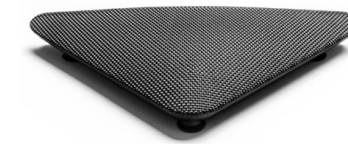





WSAUD A/S

Nymoellevej 6, DK-3540 Lynge, Denmark  
www.widex.com


Manual no.:  
9 514 0796 089 02  
Issue:  
2022-10


1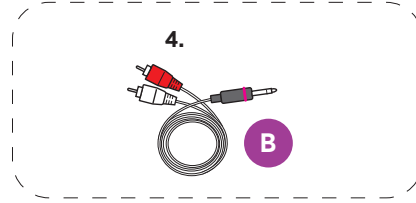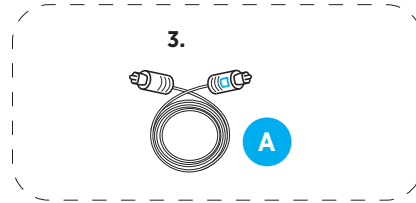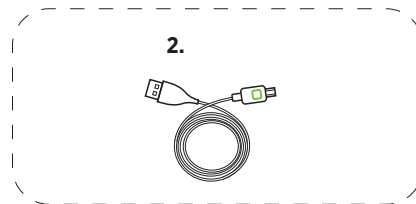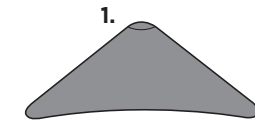


2

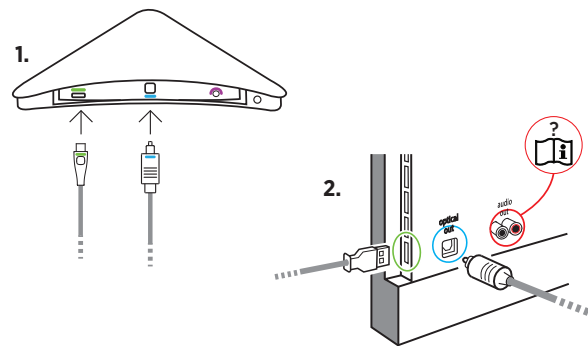
4

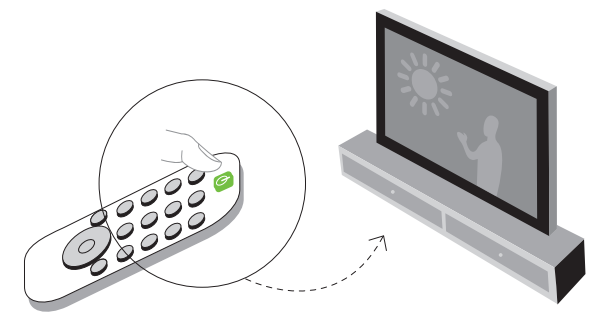
3

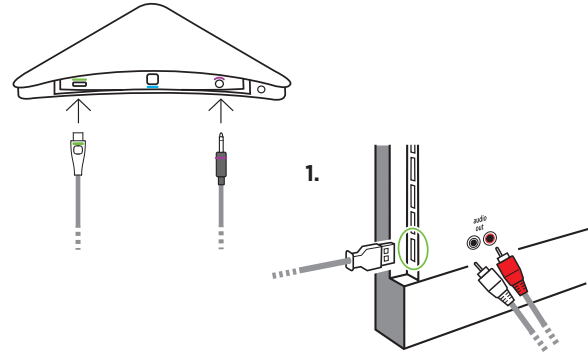5

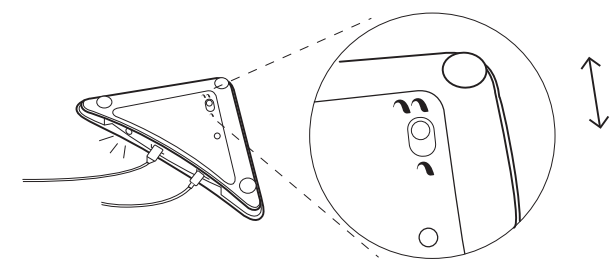
6

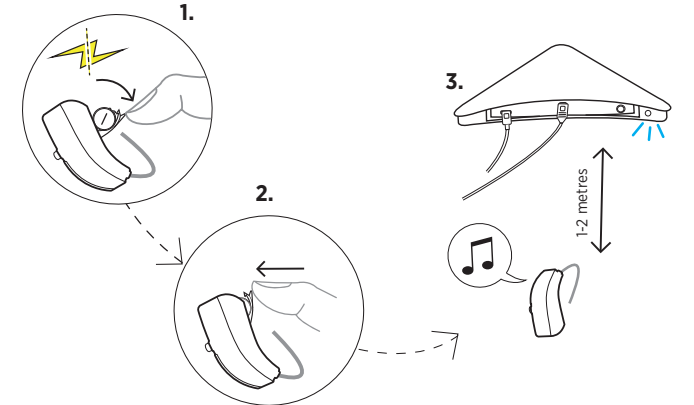7

8

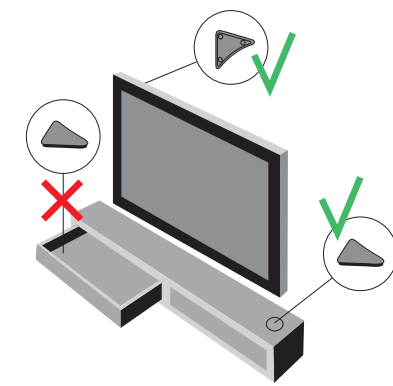SK	NÁVOD NA POUŽITIE .....	4
HU	HASZNÁLATI UTASÍTÁS .....	14
HR	KORISNIČKE UPUTE .....	24
RS	UPUTSTVO ZA UPOTREBU .....	34
SI	NAVODILA ZA UPORABO .....	44
BG	УПЪТВАНЕ ЗА УПОТРЕБА .....	54
GR	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ .....	64
LT	NAUDOJIMO INSTRUKCIJOS .....	74
RU	ИНСТРУКЦИЯ ПО ЭКСПЛУАТАЦИИ .....	84
RO	INSTRUCIUNI DE UTILIZARE .....	94
EE	KASUTUSJUHEND .....	104
BA	KORISNIČKO UPUTSTVO .....	114
LV	LIETOTĀJA INSTRUKCIJA .....	124
UA	ІНСТРУКЦІЯ З ЕКСПЛУАТАЦІЇ .....	134
AL	UDHËZIMET E PËRDORIMIT .....	144
TR	KULLANMA KILAVUZU .....	154
SA	تعليمات المستخدم .....	164
IR	راهنمای کاربر .....	174
IL	שמתשמל תוארוה .....	184
PH	MGA PAGTUTURO PARA SA USER .....	194
PK	بدايات برائے صارف .....	204
GB	USER INSTRUCTIONS .....	214
	FCC AND IC STATEMENTS .....	225
	LICENSE AGREEMENT .....	227

# NÁVOD NA POUŽITIE

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# USER INSTRUCTIONS

<b>WELCOME</b> .....	<b>215</b>
Package contents - see illustration 1.....	215
Intended use.....	215
Important safety information.....	215
<b>GETTING STARTED</b> .....	<b>216</b>
Connecting the device to your TV.....	216
Option A: using the optical cable - see illustration 2.....	216
Option B: using the analogue audio cable - see illustration 3.....	216
Powering the device.....	216
Setting up the device for one or two hearing aids - see illustration 5.....	217
Pairing the device with your hearing aids - see illustration 6.....	217
Removing a pairing.....	217
<b>USING THE DEVICE</b> .....	<b>218</b>
Using the device with the App - see illustration 7.....	218
Using the device without the App.....	218
Adjusting the volume of the TV sound.....	218
Phone calls while using the device.....	218
<b>ADDITIONAL INFORMATION</b> .....	<b>219</b>
Maintenance.....	219
LEDs.....	219
Resetting the device.....	219
Troubleshooting.....	219
<b>REGULATORY INFORMATION</b> .....	<b>222</b>
EU Directives.....	222
Directive 2014/53/EU.....	222
Information regarding disposal.....	222
<b>SYMBOLS</b> .....	<b>223</b>
Apache License (mbed TLS).....	227
Appendix: How to apply the Apache License to your work.....	230
<b>LICENSE NOTICE AND TRADEMARK ACKNOWLEDGEMENT</b> .....	<b>231</b>

# WELCOME

Congratulations on your new streaming device.

## Package contents - see illustration 1

1. Device
2. Micro USB cable
3. Optical (Toslink) audio cable
4. Analogue audio cable (mini jack-to-phon)

### NOTE

Read this booklet and the user instructions for your hearing aids carefully before you start using the device.

## Intended use

This device is intended as a wireless sound transmitter from a television to supported hearing aids.

## Important safety information

Read this section carefully before you start using the device.

Keep the device out of reach of children under three years.

Don't use the device outdoors.

Don't try to open or repair the device yourself. Contact your hearing care professional if you need to have it repaired.

Keep the device away from direct sunlight and don't place it close to any other equipment that may heat it up.

The TV or USB wall charger must comply to IEC 60601-1, IEC 60950-1, IEC 62368-1 or equivalent safety standards.

Your device should be stored and transported within the temperature and humidity ranges of -20°C to +55°C (-4°F to 131°F) and 10%-90% rH.

Your device is designed to operate within the temperature, humidity and atmospheric pressure ranges of 0°C (32°F) to 45°C (113°F), 10%-90% rH and 500 to 1500 mBar.

# GETTING STARTED

## Connecting the device to your TV

### Option A: using the optical cable - see illustration 2

For audio connection, we recommend that you use the optical cable (blue) to connect the device to your TV, as this gives you the best audio quality.

1. Remove the protection caps from both ends of the optical cable.
2. Connect the optical cable (blue) to the corresponding "Optical IN" socket (blue) on the back of the device, and then connect it to the "Optical OUT" socket on the back of your TV.

#### NOTE

Don't keep the device in a drawer - this might shorten the transmission range - see illustration 8.

#### NOTE

When using the optical cable to connect the device to your TV, the following digital formats are supported:

Digital stereo (PCM)

---

Dolby Audio™

---

It may be necessary to configure your TV, DVD player or home theater system to choose one of the supported audio formats. Notice that the device may not be able to stream sound from unsupported audio formats.

### Option B: using the analogue audio cable - see illustration 3

You can also use the analogue cable to connect the device to your TV. To do so, follow these steps:

1. Connect the plug (purple) on the analogue cable into the socket on the back of the device.
2. Then connect the analogue audio cable into the corresponding sockets on the back of your TV.

#### NOTE

Use either the optical cable option or the analogue cable option to connect your device to the TV. You don't need to use both.

#### NOTE

If you have connected both cables, the device will always choose the source from the optical connection.

## Powering the device

To power the device, connect the Micro USB cable to the device and then connect the other end to the USB socket on the back of your TV.

Then turn on your TV.

#### NOTE

If your TV doesn't have a USB socket, you can use a standard USB wall charger (not included). The wall charger should supply at least 500 mA (2.5 W) of power.

### Setting up the device for one or two hearing aids - see illustration 5

You can set up your device according to your needs using the slider on the bottom of the device. If you have one hearing aid, choose the monaural setting, and select the binaural setting if you are using two hearing aids.

Mono sound

Stereo sound

---

### Pairing the device with your hearing aids - see illustration 6

You need to pair your device and hearing aids before you start streaming sound from your TV. Do as follows:

1. Power your device by turning on your TV (see illustration 4) or consult the "Powering the device" section.
2. Restart your hearing aids by opening and closing the battery compartment.
3. Keep the hearing aids in range of the device while pairing (1-2 metres).
4. The hearing aids play a tone confirming that they have been paired with the device.

#### NOTE

Your hearing aids are only in pairing mode for three minutes after you restart them. You might need to restart them again if you didn't pair them in time.

#### NOTE

The device can be used to stream sound to more than one user at a time. If you need this option, simply follow the same pairing procedure above.

### Removing a pairing

You can only remove a pairing using the App. To do so, follow these steps:

1. Open the TV PLAY program on your App, and then enter the Sound Menu.
2. Tap the three dots on the top right corner of the Sound Menu.
3. Press the Delete pairing button and accept deleting a pairing.

## USING THE DEVICE

### Using the device with the App - see illustration 7

Once the device has been paired with your hearing aids, it will be ready to use via your App. To start listening to your TV, do as follows:

1. Start by checking that your hearing aids are paired with your phone.
2. Open your App and select the TV listening program. You'll start listening to the sound from the TV.
3. To stop listening to the sound from your TV, simply change to another listening program.

#### NOTE

The first time you use your App after pairing your hearing aids with the device, you'll see a help text on the screen informing you that a TV PLAY program has been added to your program list. This message will disappear as soon as you tap anywhere on the screen.

#### NOTE

If you don't see the TV PLAY program icon on your App, the device and hearing aids are not paired correctly.

### Using the device without the App

When the hearing aids are paired with the device, a TV program will automatically be added in the hearing aids. To stream TV sound via the TV program without using the App, you will have to toggle through the programs on your hearing aids using the program button or the RC-DEX and find the TV program.

### Adjusting the volume of the TV sound

You can easily adjust the volume of the sound from the TV or hearing aids using the volume sliders on your App screen. Without the App you can adjust the TV sound volume via the RC-DEX using the volume buttons.

#### NOTE

If you want to focus on the sound from your TV and not listen to the surrounding sounds, you can use the Room off function from your App.

### Phone calls while using the device

If you have an iPhone or Android with direct streaming, just answer incoming calls as you usually do on your phone.

#### NOTE

If the phone call is not streamed to your hearing aids, try changing to another listening program using your App.

## ADDITIONAL INFORMATION

### Maintenance

Clean your device using a soft, dry cloth, or vacuum clean it with a soft nozzle.

#### NOTE

Don't wash your device and never immerse it in water or any other liquid.

### LEDs

LED	MEANING
Fading white light	You'll see this the first time you turn on the device or after a reset
One blue flash for each hearing aid	The device is sending pairing information to your hearing aids
Fading blue light	If you make a short press on the reset button, the LED will flash blue and streaming will stop for 30 seconds. This will have no effect on your device
Fading green light	The device is on but not streaming
Constant green light	The device is on and streaming
Blinking red light	Unknown sound format
Fading red light	The device is overheated. Let the device cool down before use
	This LED pattern can also be seen if the USB cable doesn't provide the device with enough power
Constant red light	The device is not working. Contact your hearing care professional

### Resetting the device

On the back of your device you will find a button. You can use this button to reset your device by pressing it for 5 seconds. This will reboot the device and remove any existing pairings from the device.

#### NOTE

If you reset your device, you'll have to pair your hearing aids with the device again.

### Troubleshooting

These pages contain advice on what to do if your device stops working or if it doesn't work as expected. If the problem persists, contact your hearing care professional.

<b>Problem</b>	<b>Possible cause</b>	<b>Solution</b>
There is no sound from the TV in my hearing aids	<ul style="list-style-type: none"> <li>a. The hearing aids and the device are not paired</li> <li>b. The device is not connected properly to your TV</li> <li>c. The audio source is not turned on</li> <li>d. The hearing aid batteries are almost flat</li> <li>e. Your hearing aids are not in the TV PLAY program</li> <li>f. The volume is too low</li> </ul>	<ul style="list-style-type: none"> <li>a. Pair the device and your hearing aids</li> <li>b. Check that the device is properly connected to the TV</li> <li>c. Turn the audio source on</li> <li>d. Change the hearing aid batteries</li> <li>e. Select the TV PLAY program on the App</li> <li>f. Turn up the volume on the App</li> </ul>
Others can't hear the sound from the TV	<ul style="list-style-type: none"> <li>a. You have connected the device to the headphone socket</li> </ul>	<ul style="list-style-type: none"> <li>a1. Connect the analogue cable as described in the Connection chapter</li> <li>a2. Use a different output on the TV</li> <li>a3. Check the audio settings on your TV and set the loudspeakers to be used in parallel</li> </ul>
I can only hear sound in one hearing aid	<ul style="list-style-type: none"> <li>a. The battery on the other hearing aid is flat</li> <li>b. Only one hearing aid has been paired with the device</li> </ul>	<ul style="list-style-type: none"> <li>a. Change the hearing aid battery</li> <li>b. Make a new pairing</li> </ul>
I have connected the optical cable to my TV, but I have no audio in my hearing aids	<ul style="list-style-type: none"> <li>a. The TV is not configured to send audio out via the optical out connector</li> </ul>	<ul style="list-style-type: none"> <li>a. Check the TV audio output settings (advanced settings)</li> </ul>
I have connected the analogue cable to my TV, but I have no audio in my hearing aids	<ul style="list-style-type: none"> <li>The analogue connectors on the TV are only sound input</li> </ul>	<ul style="list-style-type: none"> <li>a. Use the optical cable supplied with your device</li> <li>b. Use a mini jack to mini jack cable and the headphone plug on your TV</li> <li>c. Your TV may not be compatible with the device</li> </ul>
The volume is not comfortable		<ul style="list-style-type: none"> <li>a. Adjust the volume using the App or a remote control</li> <li>b. Check the advanced settings on your TV to see volume settings</li> </ul>

<b>Problem</b>	<b>Possible cause</b>	<b>Solution</b>
The TV sound in my hearing aids is interrupted in one or both hearing aids	<ul style="list-style-type: none"> <li>a. The device has been placed in a drawer, or it is behind a wall</li> <li>b. The device is out of range</li> <li>c. There is interference from other radio transmission sources</li> </ul>	<ul style="list-style-type: none"> <li>a. Place the device in line of sight</li> <li>b. Place the device into range</li> <li>c. Move away from other radio transmission sources</li> </ul>
The LED flashes red and there is no sound in the hearing aids	<ul style="list-style-type: none"> <li>a. The device is overheated</li> <li>b. The TV cannot supply enough power to the device</li> </ul>	<ul style="list-style-type: none"> <li>a. Keep the device away from sunlight. Don't place it in a drawer and keep it away from the TV's ventilation system on the back</li> <li>b. Disconnect other equipment connected to the TV via USB, or connect the device to the power outlet</li> </ul>

If the problem persists, contact your hearing care professional.

# REGULATORY INFORMATION

## EU Directives

### Directive 2014/53/EU

Hereby, WSAUD A/S declares that this TV PLAY is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU.

The TV PLAY contains radio transmitters operating at: 2.4 GHz, 10.0 mW EIRP.

A copy of the Declaration of Conformity according to 2014/53/EU can be found at:

[www.widex.com/doc](http://www.widex.com/doc)

### Information regarding disposal

Do not dispose of hearing aids, hearing aid accessories and batteries along with ordinary household waste.

Hearing aids, batteries and hearing aid accessories should be disposed of at sites intended for electrical waste and electronic equipment, or given to your hearing care professional for safe disposal.

## SYMBOLS

Symbols commonly used by WSAUD A/S in medical device labelling (labels/IFU/etc.)

Symbol	Title/Description
	<b>Manufacturer</b> The product is produced by the manufacturer whose name and address are stated next to the symbol. If appropriate, the date of manufacture may also be stated.
	<b>Catalogue number</b> The product's catalogue (item) number.
	<b>Consult instructions for use</b> The user instructions contain important cautionary information (warnings/precautions) and must be read before using the product.
	<b>Warning</b> Text marked with a warning symbol must be read before using the product.
	<b>WEEE mark</b> "Not for general waste". When a product is to be discarded, it must be sent to a designated collection point for recycling and recovering to prevent the risk of harm to the environment or human health as a result of the presence of hazardous substances.
	<b>CE mark</b> The product is in conformity with the requirements set out in European CE marking directives.
	<b>RCM mark</b> The product complies with electrical safety, EMC and radio spectrum regulatory requirements for products supplied to the Australian or New Zealand markets.
	<b>Medical device</b> Indication that the device is a medical device.

# CONTENTS

## FCC AND IC STATEMENTS

FCC ID: TTY-TVP

IC: 5676B-TVP

CAN ICES-3(B)/NMB-3 (B)

Federal Communications Commission Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Changes or modifications to the equipment not expressly approved by WSAUD A/S could void the user's authority to operate the equipment.

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Under Industry Canada regulations, this radio transmitter may only operate using an antenna of a type and maximum (or lesser) gain approved for the transmitter by Industry Canada.

To reduce potential radio interference to other users, the antenna type and its gain should be so chosen that the equivalent isotropically radiated power (e.i.r.p.) is not more than that necessary for successful communication.

This device complies with Industry Canada licence-exempt RSS standard(s). Operation is subject to the following two conditions:

- (1) this device may not cause interference, and
- (2) this device must accept any interference, including interference that may cause undesired operation of the device.

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Version 2.0, January 2004

<http://apache.org/licences/>

### TERMS AND CONDITIONS FOR USE, REPRODUCTION, AND DISTRIBUTION

#### 1. Definitions.

"License" shall mean the terms and conditions for use, reproduction, and distribution as defined by Sections 1 through 9 of this document.

"Licensor" shall mean the copyright owner or entity authorized by the copyright owner that is granting the License.

"Legal Entity" shall mean the union of the acting entity and all other entities that control, are controlled by, or are under common control with that entity. For the purposes of this definition, "control" means (i) the power, direct or indirect, to cause the direction or management of such entity, whether by contract or otherwise, or (ii) ownership of fifty percent (50%) or more of the outstanding shares, or (iii) beneficial ownership of such entity.

"You" (or "Your") shall mean an individual or Legal Entity exercising permissions granted by this License.

"Source" form shall mean the preferred form for making modifications, including but not limited to software source code, documentation source, and configuration files.

"Object" form shall mean any form resulting from mechanical transformation or translation of a Source form, including but not limited to compiled object code, generated documentation, and conversions to other media types.

"Work" shall mean the work of authorship, whether in Source or Object form, made available under the License, as indicated by a copyright notice that is included in or attached to the work (an example is provided in the Appendix below).

"Derivative Works" shall mean any work, whether in Source or Object form, that is based on (or derived from) the Work and for which the editorial revisions, annotations, elaborations, or other modifications represent, as a whole, an original work of authorship. For the purposes of this License, Derivative Works shall not include works that remain separable from, or merely link (or bind by name) to the interfaces of, the Work and Derivative Works thereof.

"Contribution" shall mean any work of authorship, including the original version of the Work and any modifications or additions to that Work or Derivative Works thereof, that is intentionally submitted to Licensor for inclusion in the Work by the copyright owner or by an individual or Legal Entity authorized to submit on behalf of the copyright owner. For the purposes of this definition, "submitted" means any form of electronic, verbal, or written communication sent to the Licensor or its representatives, including but not limited to communication on electronic mailing lists, source code control systems, and issue tracking systems that are managed by, or on behalf of,

the Licensor for the purpose of discussing and improving the Work, but excluding communication that is conspicuously marked or otherwise designated in writing by the copyright owner as "Not a Contribution."

"Contributor" shall mean Licensor and any individual or Legal Entity on behalf of whom a Contribution has been received by Licensor and subsequently incorporated within the Work.

## **2. Grant of Copyright License.**

Subject to the terms and conditions of this License, each Contributor hereby grants to You a perpetual, worldwide, non-exclusive, no-charge, royalty-free, irrevocable copyright license to reproduce, prepare Derivative Works of, publicly display, publicly perform, sublicense, and distribute the Work and such Derivative Works in Source or Object form.

## **3. Grant of Patent License.**

Subject to the terms and conditions of this License, each Contributor hereby grants to You a perpetual, worldwide, non-exclusive, no-charge, royalty-free, irrevocable (except as stated in this section) patent license to make, have made, use, offer to sell, sell, import, and otherwise transfer the Work, where such license applies only to those patent claims licensable by such Contributor that are necessarily infringed by their Contribution(s) alone or by combination of their Contribution(s) with the Work to which such Contribution(s) was submitted. If You institute patent litigation against any entity (including a cross-claim or counterclaim in a lawsuit) alleging that the Work or a Contribution incorporated within the Work constitutes direct or contributory patent infringement, then any patent licenses granted to You under this License for that Work shall terminate as of the date such litigation is filed.

## **4. Redistribution.**

You may reproduce and distribute copies of the Work or Derivative Works thereof in any medium, with or without modifications, and in Source or Object form, provided that You meet the following conditions:

1. You must give any other recipients of the Work or Derivative Works a copy of this License; and
2. You must cause any modified files to carry prominent notices stating that You changed the files; and
3. You must retain, in the Source form of any Derivative Works that You distribute, all copyright, patent, trademark, and attribution notices from the Source form of the Work, excluding those notices that do not pertain to any part of the Derivative Works; and
4. If the Work includes a "NOTICE" text file as part of its distribution, then any Derivative Works that You distribute must include a readable copy of the attribution notices contained within such NOTICE file, excluding those notices that do not pertain to any part of the Derivative Works, in at least one of the following places: within a NOTICE text file distributed as part of the Derivative Works; within the Source form or documentation, if provided along with the Derivative Works; or, within a display generated by the Derivative Works, if and wherever such third-party notices normally appear. The contents of the NOTICE file are for informational purposes only and do not modify the License. You may add Your

own attribution notices within Derivative Works that You distribute, alongside or as an addendum to the NOTICE text from the Work, provided that such additional attribution notices cannot be construed as modifying the License.

You may add Your own copyright statement to Your modifications and may provide additional or different license terms and conditions for use, reproduction, or distribution of Your modifications, or for any such Derivative Works as a whole, provided Your use, reproduction, and distribution of the Work otherwise complies with the conditions stated in this License.

#### **5. Submission of Contributions.**

Unless You explicitly state otherwise, any Contribution intentionally submitted for inclusion in the Work by You to the Licensor shall be under the terms and conditions of this License, without any additional terms or conditions. Notwithstanding the above, nothing herein shall supersede or modify the terms of any separate license agreement you may have executed with Licensor regarding such Contributions.

#### **6. Trademarks.**

This License does not grant permission to use the trade names, trademarks, service marks, or product names of the Licensor, except as required for reasonable and customary use in describing the origin of the Work and reproducing the content of the NOTICE file.

#### **7. Disclaimer of Warranty.**

Unless required by applicable law or agreed to in writing, Licensor provides the Work (and each Contributor provides its Contributions) on an "AS IS" BASIS, WITHOUT WARRANTIES OR CONDITIONS OF ANY KIND, either express or implied, including, without limitation, any warranties or conditions of TITLE, NON-INFRINGEMENT, MERCHANTABILITY, or FITNESS FOR A PARTICULAR PURPOSE. You are solely responsible for determining the appropriateness of using or redistributing the Work and assume any risks associated with Your exercise of permissions under this License.

#### **8. Limitation of Liability.**

In no event and under no legal theory, whether in tort (including negligence), contract, or otherwise, unless required by applicable law (such as deliberate and grossly negligent acts) or agreed to in writing, shall any Contributor be liable to You for damages, including any direct, indirect, special, incidental, or consequential damages of any character arising as a result of this License or out of the use or inability to use the Work (including but not limited to damages for loss of goodwill, work stoppage, computer failure or malfunction, or any and all other commercial damages or losses), even if such Contributor has been advised of the possibility of such damages.

#### **9. Accepting Warranty or Additional Liability.**

While redistributing the Work or Derivative Works thereof, You may choose to offer, and charge a fee for, acceptance of support, warranty, indemnity, or other liability obligations and/or rights consistent with this License. However, in accepting such obligations, You may act only on Your own behalf and on Your

sole responsibility, not on behalf of any other Contributor, and only if You agree to indemnify, defend, and hold each Contributor harmless for any liability incurred by, or claims asserted against, such Contributor by reason of your accepting any such warranty or additional liability.

END OF TERMS AND CONDITIONS

### **Appendix: How to apply the Apache License to your work**

To apply the Apache License to your work, attach the following boilerplate notice, with the fields enclosed by brackets "[ ]" replaced with your own identifying information. (Don't include the brackets!) The text should be enclosed in the appropriate comment syntax for the file format. We also recommend that a file or class name and description of purpose be included on the same "printed page" as the copyright notice for easier identification within third-party archives.

Copyright [yyyy] [name of copyright owner]

Licensed under the Apache License, Version 2.0 (the "License"); you may not use this file except in compliance with the License. You may obtain a copy of the License at <http://www.apache.org/licenses/LICENSE-2.0>

Unless required by applicable law or agreed to in writing, software distributed under the License is distributed on an "AS IS" BASIS, WITHOUT WARRANTIES OR CONDITIONS OF ANY KIND, either express or implied. See the License for the specific language governing permissions and limitations under the License.

Copyright © Widex A/S, 2017 - 2018. All rights reserved. Any unauthorized redistribution or use of the software in source or binary form is prohibited and will be prosecuted when discovered.

## **LICENSE NOTICE AND TRADEMARK ACKNOWLEDGEMENT**

Manufactured under license from Dolby Laboratories. Dolby, Dolby Audio and the double-D symbol are trademarks of Dolby Laboratories.